ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE ENTIRE INSTRUCTION MANUAL.

CORRECT APPLICATION AND CARE IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE / INDICATIONS: The Wrist Brace (WB) with Boa® provides stabilization to the wrist joint.

CONTRAINDICATIONS: Patients with known allergies to foams, adhesives, or plastic products.

WARNINGS AND PRECAUTIONS:

Instruction for wearing, opening, closing or removing the brace should be provided by a healthcare professional.

Brace should be applied by a qualified healthcare professional.

Do not heat brace on the body.

Check to make sure brace is not too hot prior to application.

Brace should be appropriately sized to the patient to ensure proper support and function.

Care should be taken with patients with delicate or compromised skin, especially the elderly, not to apply the brace too tightly, which may cause pressure and shear to the fragile tissue. PATIENTS SHOULD BE INSTRUCTED ON APPROPRIATE USE AND LONG-TERM ONGOING USE. An under sleeve or stockinette is used to pad the product.

Do not heat Exos® products to greater than 120°F (100°C).

If the brace does not fit properly, or is irritating, it may be reformed and adjusted by a healthcare professional.

If patient experiences any pain, swelling, sensation changes, or any unusual reactions while using this product, consult a healthcare professional immediately.

Patient should be instructed to check for breaks and cracks and informed to see a healthcare professional for a replacement.

Patient should be instructed to never clean and drying instructions (see Patient Cleaning Instructions below).

Patient should be instructed not to expose brace to heat over 130°F (55°C) as the brace may soften and lose its desired shape.

Patient should be instructed not to expose the brace to moisturizers, chemicals, or solvents, as they may affect durability.

If the brace is applied with a Locking Ring, the patient must not get brace wet.

OPERATING BOA® LACING SYSTEM:

To Loosen Boa®:

- Lift Boa® Knob. (Figure 1)

- Pull Tab. (Figure 2)

To Close and Tighten Boa®:

- Push Boa® Knob down. (Figure 3)

- Boa® Knob must be in the down locked position when wearing brace.

HEATING AND APPLICATION:

- Open the WB. (Figure 4)

- Heat brace to 100°F (38°C).

- Heat brace in Exos® recommended oven approximately 3-4 minutes or until brace is pliable. Brace should not be too hot to handle comfortably.

- Ensure that skin is completely dry before application.

- Fit brace on patient. (Figure 5)

- Remove slack face by tightening Boa® Knob.

Caution: Do not tighten brace fully until brace has returned to room temperature otherwise permanent deformation may occur.

- Hook-and-loop fastener strap may be secured in the desired position. (Figure 7)

Once brace has hardened, the Boa® Knob may be tightened to desired tension.

Warning: The brace should not be worn too tightly. This may produce excessive mechanical force or shear forces to the skin. (Figure 8)

PATIENT CLEANING AND STORAGE INSTRUCTIONS:

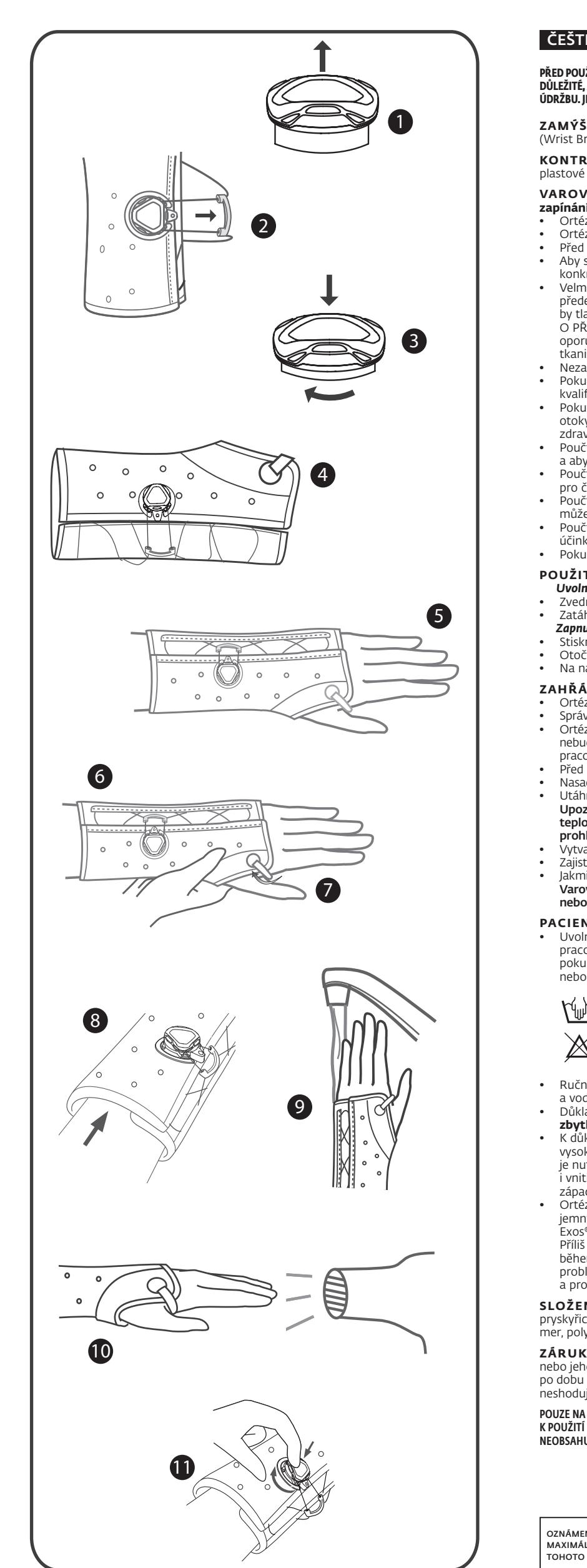
- Loosen or remove the brace as instructed by healthcare professional. The brace may be washed or worn while swimming or bathing, only if the healthcare professional allows the patient to loosen or remove the brace. If not, the patient cannot get the brace and skin wet.

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