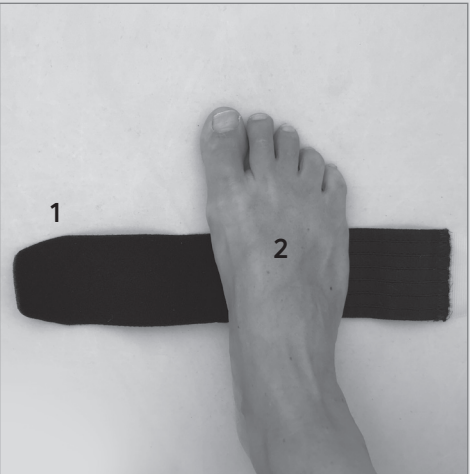
DEUTSCH

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Polyamid, Polyurethan, Nylon, Elastan, Polyester, SBR-Schaum

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ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

The mid-foot brace is designed to lift the mid-foot bone and support transverse arch so providing pain relief for conditions such as:

- Hallux Valgus
- Splay foot, flat foot (fallen arch)
- Metatarsalgia

Performance characteristics:

Providing mild protection of a limb or body segment for acute and prophylactic (preventive) care.

CONTRAINDICATIONS

- If you have the following conditions please consult a physician before using the device: gout attack, peripheral arterial disease or impaired blood circulation, rheumatic diseases, hypersensitivity to pressure, base joint arthritis, polyneuropathy or soft tissue swelling.
- Do not use this device if you have skin diseases or injuries in the affected area of the body.
- Any known allergies to the medical devices materials.

WARNING & PRECAUTIONS

The product is intended for single patient use only. The product must be worn directly on the skin. Do not apply any lotions or ointments on the relevant part of the body while wearing the support. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact a medical professional immediately. Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

The mid-foot brace can be worn on the left or right foot and fits all foot sizes.

- 1) Undo straps to open the brace.
- 2) Place the foot over the pad and adjust as necessary. Ensure that the pad is fitted in the arch of the foot to lift the base of the metatarsal bones (bones of the forefoot). The pad should not extend the line of the ball of foot and should be comfortable while walking.
- 3) Place left strap on the top of the foot.
- 4) With your left hand, hold left strap in place and, with your right hand pull right strap to the desired tightness and secure end to left strap. Loosen the straps or remove the brace immediately in the event of tingling, bruising, swelling, or numbness.

Note: The brace fits in shoes with a wide front opening. Shoes will also need to be wide enough to accommodate the brace.

CLEANING INSTRUCTIONS

Machine washable at gentle cycle (max 30°C) with a mild detergent. Hook and loop of the straps should be closed for washing to avoid any damage.

COMPOSITION

Polyamide, Polyurethane, Nylon, Elastane, Polyester, SBR Foam

NOT MADE WITH NATURAL RUBBER LATEX

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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