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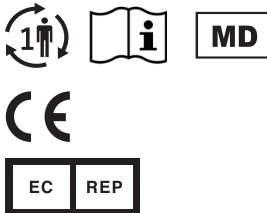
DONJOY® HINGED AIR
KNEE BRACE
DONJOY®

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DJO, LLC
5919 Sea Otter Place,
Suite 200
Carlsbad, CA 92010
USA

13-2066-0-00000 REV D 2021-12-31

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

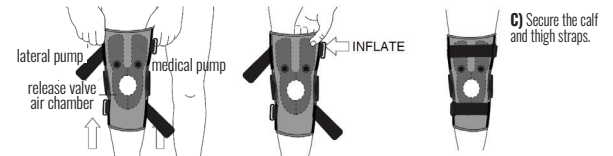
INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

The DonJoy Drytex® Hinged Air® Knee Brace is designed to provide support to the knee. It may be suitable for use following a knee injury, soft-tissue injury and medial/lateral instabilities. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.


CONTRAINDICATIONS: None.

A) While seated on the edge of a chair open back panel of brace and step through the brace. Slide brace over the knee so that the top of the circular cutout aligns with the top of the kneecap, and hinges are parallel to each other. Close back panel of brace so it feels snug but not too tight.

B) To inflate the medial (towards midline of body) or lateral (away from midline of body) air chamber, slip hand under the brace and press pump with thumb. Pump until desired inflation is achieved. **Note:** Once the air chamber has been inflated, it is not necessary to deflate it to remove the brace. The pad can remain inflated for repeated wearings and deflated when desired.

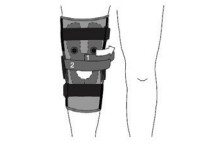D) Deflate the brace by pressing the deflate button.


F) For lateral (towards midline of body) tilt correction; subluxation (partial dislocation of the knee) and dislocation; inflate lateral (towards midline of body) buttress. Attach short strap on the top inside corner of the cutout hole. (1) Pull the strap firmly towards the inside of the leg and anchor it to the inside loop material. Next, attach long strap on the outside loop material. (2) pull the strap firmly towards the inside of the leg and anchor it to the inside loop material. **Note:** For medial (away from midline of body) tilt, subluxation (partial dislocation of the knee) and dislocation, reverse the instruction.

E) For lateral (towards midline of body) tilt correction; inflate lateral (towards midline of body) buttress. Attach short strap on medial (away from midline of body) buttress and attach to medial away from midline of body loop. **Note:** Strap sequence can be modified for individual patient needs.

E) For external rotation; Locate the long strap with the hook material in the center of the strap. (1) inflate (towards midline of body) buttress and attach short strap under the knee cap. Pull the strap firmly towards the inside of the leg and anchor it to the inside loop material. Attach the long strap midway and outside the knee cap. (2) pull the long strap firmly towards the inside of the leg and anchor it to the inside loop material. **Note:** For internal rotation, reverse the instruction.

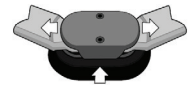HINGE ADJUSTMENT: A 10° extension stop is installed in the brace due to soft tissue compression. Remove the screws with a Phillips head screwdriver. It is necessary to slide the protective shield cover off before removing the stops. **Note:** Both medial (away from midline of body) and lateral (towards midline of body) stops must be identical.

USE AND CARE: The brace is compatible with fresh or salt water. Following use in water activities, drain thoroughly and rinse with clear water. AIR DRY. Do not depress release valves when brace is submerged in water. Hand wash in cold water with mild soap. Rinse thoroughly. Air dry. **Note:** If not rinsed thoroughly, residual soap may cause skin irritation and deterioration of the material. The hinge mechanism requires little maintenance in normal use. However, you may apply a dry lubricant such as Teflon® spray to the internal mechanism of the hinge. Dry lubrication is recommended following use in water. Periodically inspect hinge screws for tightness and retighten as needed. Always consult with your physician or therapist before making changes to the brace.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

WARNINGS AND PRECAUTIONS: It is recommended to have the device fitted by a suitably qualified healthcare professional. If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. Do not use over open wounds. Do not use this device if it was damaged and/or packaging has been opened. **NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

FOR SINGLE PATIENT USE ONLY.

NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2021 DJO, LLC
MDSS GmbH