## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE ENTIRE INSTRUCTION MANUAL.**  
**CORRECT APPLICATION AND CARE IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:**

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INTENDED USE / INDICATIONS:** The Radial Gutter Fracture Brace (RGFB) is designed to stabilize the second and third metacarpals, second and third digits and the second and third metacarpal/garlic joint, and the wrist joint.

**PERFORMANCE CHARACTERISTICS:** Providing stabilization to the limb or body segment. Providing immobilization or controlled movement to the limb or body segment.

**CONTRAINDICATIONS:** Do not use if you are allergic to any of the materials contained within this product.

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**WARNINGS and PRECAUTIONS:** **Instruction for wearing, opening, closing or removing the brace should be provided by a healthcare professional.**

This product must be prescribed and fitted by a healthcare professional. The frequency and duration of use should be determined by your healthcare professional. • Do not heat the brace on the body. • Check to make sure brace is not too hot prior to application.

• Brace should be appropriately sized to the patient to ensure proper function. • Wearng brace should be taken with patients with delicate or compromised skin, especially those who have had recent surgery or skin grafts. • Use should be limited to a maximum of 100 °F (38 °C). • Do not apply brace to areas of the body that are sensitive to heat.

• Do not use brace if it is uncomfortable or irritating. • If brace does not fit properly, or is irritating, it may be reformed and adjusted by a healthcare professional. • Do not use device other than instructed in the IFU. • Do not modify the device without a healthcare professional present. • If patient experiences any pain, swelling, sensation changes, or any unusual reactions while using this product, consult a healthcare professional immediately. • Patient should be instructed to check for breaks and cracks and informed to see a healthcare professional for a replacement. • Patient should be instructed to clean and dry the brace before and after use. • Do not apply brace to areas of the body that are sensitive to heat.

• Patient should be instructed on proper cleaning and drying instructions (see Patient Cleaning Instructions below). • If the brace is applied with a locking Ring, the patient must not get brace wet. • The patient should be instructed not to expose brace to heat over 130°F (55°C) as the brace may soften and lose its desired shape. • Keep out of reach of children under five years of age to avoid accidental ingestion of the small parts. • Patient should be instructed not to expose brace to moisture, chemicals, or solvents, as they may affect durability. • Do not use open wounds. • Do not use the device if it is damaged or off the packaging has been opened.

• NOTE: Contact manufacturer and competent authority in case of a serious incident during use of this device.

**OPERATION BOA® LACING SYSTEM:**

To open:

- Lift Boa® Knob. (Figure 1)

- Tie the lengueta. (Figure 2)

- Pull hook-and-loop Release Ribbon - to fully open.

To close:

- Push Boa® Knob down. Turn Boa® Knob clockwise. (Figure 3)

- Boa® Knob should be in the down/locked position when wearing brace.

**HEATING AND APPLICATION:** Detach hook-and-loop fastener Full Release Ribbon and pull open Boa® lace. (Figure 4)

**Note:** Larger braces are packaged with the finger sections folded lengthwise. Do not fold fingers until the brace has been fully heated.

Proper heating temperature is 200°F (93°C).

Heat brace in Exos® recommended oven approximately 5 minutes or until brace is pliable. Brace should not be too hot to handle comfortably.

Ensure brace is completely dry before application.

Fit brace on patient. (Figure 5)

Secure hook-and-loop fastener Full Release Ribbon.

Secure hook-and-loop fastener straps on fingers.

Secure strap to the tightener until brace has returned to room temperature otherwise indentations in plastic may occur.

Mold brace into desired position. (Figure 6)

Once brace has hardened, the Boa® Reel may be tightened to desired tension.

Warning: Tighten Boa® Reel may produce excessive mechanical force or shear forces to the skin. (Figure 7)

Patient CLEANING AND STORAGE INSTRUCTIONS:

- Loosen or remove the brace as instructed by healthcare professional. The brace may be washed or worn while swimming or bathing, only if the healthcare professional allows the patient to loosen or remove the brace.

Proper cleaning and storage instructions:

- Properly wash and dry the brace.

- Do not expose brace to heat over 130°F (55°C).

- Do not expose to water. NOTE: If not washed thoroughly, residual soap may cause irritation and could deteriorate material.

- Use a hair dryer set on high volume and cool setting to thoroughly dry the interior of the brace and skin. (Figure 9) It is imperative that the inside surface of the brace and skin is clean and dry before use.

- Retighten the brace. (Figure 10) Exos® Braces are meant to be worn under light pressure and not tight to the skin. A little wiggle room inside the Exos® brace allows for a more comfortable fit. Patients who have had a cast removed from their arm will experience shear or pressure on the skin during motion, and can result in irritation, rash, odor, or skin issues. Wearing the brace too loose will reduce support and may compromise healing.

**MATERIAL CONTENTS:** EVA foam, spandex, polyester, steel, nylon 6/6 resin, antimicrobial treatment, polycarbonate, thermoplastic elastomer, polyoxymethylene.

**COMPOSITION OF THE MATERIALS:** Espuma EVA, elástano, nailon, terpolímero, poliéster, acero, resina de nailon 6/6, tratamiento antimicrobiano, poliuretano, poliuretano teroplástico, polioximetileno.

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