ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER

The intended user should be a licensed healthcare professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS

The Donley Inmestrap 2.0 is designed to provide comfort and support of the lumbar spine. It may be suitable for the symptomatic relief of acute and chronic low back pain. Provides compression without significant restriction of mobility, can be adjusted as and when required by the adjustable straps.

- Acute lower back pain
- Chronic low back pain
- Muscular weakness
- Lumboscoliosis
- Lumbalgia
- Lumbar arthritis
- Spondylolisthesis
- Herniated disc
- Facet syndrome
- Sacralgia symptoms

PERFORMANCE CHARACTERISTICS

- Providing support and stabilization to the limb or body segment.
- Providing partial immobilization or controlled movement to the limb or body segment.
- Providing limited restriction of a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not use if you are allergic to any of the materials contained within this product.

WARNING AND PRECAUTIONS

- We recommend that the first application be made under supervision of a healthcare professional.
- If you have skin conditions/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a healthcare professional.
- If you experience any discomfort, pain, swelling or sensation changes, or any unusual reactions while wearing the product, consult your healthcare professional immediately.
- This product is for single patient multiple use only.
- Do not use this device if it was damaged prior to packaging has been opened.
- The product should be removed during long periods of rest.
- Do not use during showering or bathing.
- Keep the product away from children.
- Do not use on open wounds/s) and/or sensitive skin.
- Do not use this product if any of the parts are damaged.
- Do not modify the device.
- Do not use this product in any way that may damage its integrity or use the device other than intended.
- The frequency and duration of use should be determined by a healthcare professional.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

FOR SINGLE PATIENT USE ONLY.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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APPLICATION INSTRUCTIONS

Ensure that the device is correctly fitted around the back support. If possible, the back support should be put on whilst standing. Open the belt and loosen the 4 lateral straps.

- Place the belt around the waist, placing your hands in the finger loops located on the front closures. Make sure that the lumbar belt is correctly centred over your spine (the stays should be positioned on either side of the spine). Position the bottom part of the belt at the top of the gluteal cleft.
- Pull the finger loops to extend the main fabric and wrap it around your waist.
- Fasten the belt with the right side over the left.
- 4a/b Pull the lateral straps forward equally with gentle force until the desired level of support is achieved. Then fasten them to the front closure using the hook-and-loop fasteners.
- The 2 front straps can be removed through the slots.
- The level of compression can be readjusted as needed during the day.

ACCESSOIRES

The Donley Inmestrap 2.0 has been designed to be used with the following optional accessories:

- Massage pack pad. The dots should face the back's curvature. Fasten the pad to the back of the belt using the hook-and-loop fasteners (arrow symbol pointed upwards).
- Shoulder straps. These allow you to open the belt when support is not needed without having to remove the entire belt. Attach the shoulder straps to the belt using the hook-and-loop fasteners.
- Hot/cold pack with pouch. Fasten the pouch to the back belt using the hook-and-loop fasteners.

CLEANING INSTRUCTIONS

Machine washable at gentle cycle (max 40°C) with a mild detergent. Hook-and-loop fasteners of the straps should be closed for washing to avoid any damage. Do not bleach. Rose throughly. Air dry. Do not use machine dry. Do not iron. Do not touch.

DISPOSAL

Dispose of the device and its packaging in accordance with local or national environmental regulations.

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