## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### INTENDED USE/INDICATIONS

- Support and compression following acute and chronic knee injury and mild knee osteoarthritis
- Proprioceptive and neuromuscular stimulation through compression (to support balance, conditions and movements)
- Mild to moderate ligament sprains
- Mild to moderate knee instability
- Mild meniscus injuries
- Prevention of injury

### Performance characteristics:

Provides mild compression of limb or body segment through elastic construction.

Provides mild protection of a limb or body segment for acute and prophylactic care.

### CONTRAINDICATIONS

- Any known allergies to the medical device materials.
- Peripheral arterial disease.
- Impaired blood flow in the leg (e.g. with diabetes mellitus).
- Impaired lymphatic drainage or soft tissue swelling of unknown origin.

### WARNINGS AND PRECAUTIONS

- The product is intended for single patient use only
- If you have skin conditions/trauma in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not apply on open wounds.
- The product is designed to be worn directly on the skin.
- Do not apply any lotions or ointments on the relevant part of the body while wearing the support.
- Remove the support during longer periods of rest.
- If patients experience pain, swelling, changes in sensation or other unusual reactions occur while using this product, they should contact their doctor immediately.
- Do not use this device if it is damaged and/or packaging has been opened.
- Do not use this product in any way that may damage its integrity.
- Handle with care: avoid fastening the strap's hook closure onto the knitted sleeve as it may cause damage to the appearance of the product when strap is removed.

**NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.**

### APPLICATION INSTRUCTIONS

- Sit down to fit the Genuforce Xpert.
- Unfasten and loosen the two straps at the thigh and lower leg without pulling them out of the slits.
- Grip edges of the support at the larger opening.
- Pull the support over the knee.
- Adjust the fit of the support as necessary.
- Ensure that the circular pad encloses the kneecap and that the hinges are parallel to each other.
- Tighten and secure the straps to stabilize the knee comfortably.

### Removing the bilateral hinges:

- Undo the straps and remove them from the hinges.
- Take the hinges out of the covers through the slits on the support.

### Replacing the bilateral hinges and the straps:

- Insert the hinges in the covers through the slits on the support.
- Reattach the straps onto the hinges.

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### COMPOSITION

Elastane, Nylon, Polyester, Polyolefin, Polyamide, Cotton, Polyurethane, Elastomer, Silicone, Elastane, Glass fiber

### NOT MADE WITH NATURAL RUBBER LATEX

### FOR SINGLE PATIENT USE ONLY.

### WARRANTY D.J.O, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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