## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### INTENDED USE/INDICATIONS:

The DonJoy Fusiolight is designed to support the ankle. It may be suitable for use following mild to moderate ankle sprains, and help with Chronic Ankle Instabilities (CAI). Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

### PERFORMANCE CHARACTERISTICS:

Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

### CONTRAINDICATIONS:

- Do not use if you are allergic to any of the materials contained in this product.

### WARNINGS AND PRECAUTIONS:

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INSTRUCTIONS:

- Place the foam liners pads inside the rigid bilateral shells. (Fig. 1)
- Adjust the heel strap for proper fit. (Fig. 2)
- Pull the outer shells and foam liners along the ankle, making sure the malleoli are well protected. (Fig. 3)
- Make sure the brace fits snugly, and then close the brace using the contact closure straps. (Fig. 4)
- If necessary, adjust the heel pad with the vertical side straps while the brace is on. Pull straps upward until desired fit is obtained. (Fig. 5)
- For maximum comfort and support, wear a long absorbent sock and a laced athletic shoe when wearing the brace. (Fig. 6)

### CARE:

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not iron.

### COMPOSITION:

Polyester, polyurethane, polyamide, silicone, polypropylene

**FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.**

**WARRANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOTICE:** WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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