ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:
The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to:
• Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:
The Donjoy IMMO AT4 is designed to provide support and immobilization to the knee. It may be suitable for use following a knee injury or soft tissue injury. Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS:
Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS:
1) It is recommended to have the device fitted by a suitably qualified healthcare professional.
2) If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
3) Do not use over open wounds.
4) Do not use this device if it was damaged and/or package has been opened.
NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.
CAUTION: This brace is intended to be used for immobilization of a patient's lower limb following injury or surgery. It is NOT intended to support the full body weight of a patient while standing or walking.

APPLICATION INSTRUCTIONS:
1) With the patient lying down, open-up the brace and place it under the patient's extended leg. Adjust the internal anti-slip strap, under the patella. (Fig. 1)
2) Wrap the brace over the leg, taking care to place the patella in the front opening of the brace. (Fig. 2)
3) Close the immobilizer and tighten the closure straps. Once fitted, the immobilizer can be easily put into place and removed by using the buckles. Tighten the straps according to the circumference of the thigh and calf, making sure they are neither too tight nor too loose. (Fig. 3)
The posterior and lateral stays can be shaped according to the patient's needs. Remove the stays from their casings (open the contact closure tab at the top of the casing and pull the stay out). Bend them to the required angle then place them back in their casings.

CARE:
Wash in lukewarm water (30°C) with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION:
Polyester, Polyurethane, Polyamide, Synthetic Resin, Elastane, Polypropylene, Aluminium, Polyethersulfone

FOR SINGLE PATIENT USE ONLY.
LATEX
CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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