**ENGLISH**

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The user should be able to:

- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

This device can be used for patients with:

- Ankle sprains
- Chronic ankle instability
- Ankle injury rehabilitation/immobilization
- Prophylactic use
- Prevention or treatment of acute ankle sprains

PERFORMANCE CHARACTERISTICS:

Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

WARNINGS AND PRECAUTIONS:

Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene byproducts. Do not use over open wounds. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. Do not use this device if it was damaged and/or package has been opened.

Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

ANKLE BRACE SHOULD BE WORN WITH AN ATHLETIC SOCK.**APPLICATION INFORMATION:**

1. Before application, loosen straps and laces. The ankle must be at a 90° angle when applying the support. While seated at the edge of a chair, place foot in ankle support. Make sure neoprene tongue of support is on the top of the foot area prior to lacing-up. Lace up the support like a shoe.
2. For application to the right foot, turn heel out slightly. For left foot, turn heel in slightly. Starting with strap "A", wrap over forefoot and then under the arch area of the foot. Secure with tension to pressure-sensitive closures. Trim strap "A" if necessary.
3. For right foot, rotate heel slightly inward. For left foot, rotate heel slightly outward. Apply strap "B" over forefoot and then under arch area of foot. Secure with tension to pressure-sensitive closure. Trim strap "B" if necessary.
4. Wrap circumferential strap around ankle over straps "A" and "B" to secure to itself.
Note: Ankle support should be snug but not impair circulation.

CLEANING INSTRUCTIONS:

Hand wash with mild soap in lukewarm water. Rinse thoroughly. AIR DRY.

Note: If not rinsed thoroughly, residual soap may cause skin irritation or deteriorate the material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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