MSS GmbH
Söflinger Str. 17
30775 Hannover, Germany
V193 49 20281-5553 -USA
DJO LLC
180 Decision Street
Port Decision, USA

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver... INDICATIONS: The DonJoy Defiance ACL, PCL, CI Knee Brace is designed to protect and stabilize the knee...

- A) While sitting on the edge of a chair, step through the brace. Flexing the knee to 45°, position the brace as shown in figure 1... B) Secure the bottom calf strap first (#1), then the top thigh strap (#2)...

INSTRUCTIONS FOR HINGE ADJUSTMENT: Your brace comes with 10" extension stops installed. To change the installed stops, first completely remove the screw as shown (figure 5)...

USE AND CARE: If during activity, re-tightening is necessary, re-tighten all straps following steps in the application instructions... Do not use this device if it is damaged and/or the packaging has been opened.

WARNINGS & PRECAUTIONS: This product must be prescribed and fitted by a healthcare professional. The frequency, duration of use and directions for use should be determined by your healthcare professional... Do not use this device if it is damaged and/or the packaging has been opened.

WARNING: DJO, LLC will repair or replace all or part of the product and its accessories for material or workmanship defects for a period of six months from the date of purchase... Do not use this device if it is damaged and/or the packaging has been opened.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION...

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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