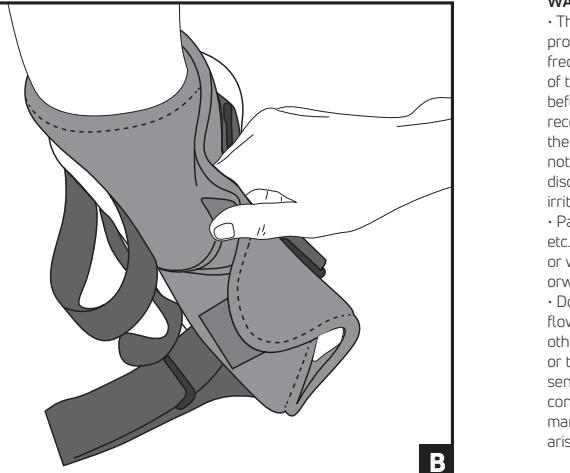
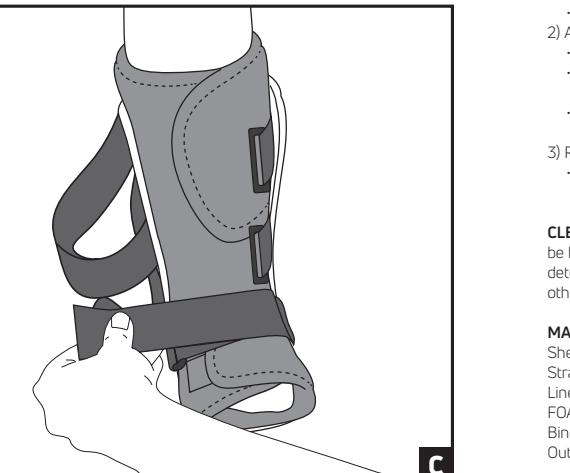
**A****B****C****ENGLISH**

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INDICATIONS:** The Procure MiniTrax Walker is designed to provide support and immobilization to the lower leg, ankle and foot. It may be suitable for the treatment of stable foot fractures, ligamentous injuries such as Plantar Fasciitis, tendon injuries such as Achilles tendon repair and lateral ankle sprains.

**PERFORMANCE CHARACTERISTICS:** Providing immobilization or controlled movement of the limb or body segment.

**CONTRAINDICATIONS:** Do not use if you are allergic to any of the materials contained in this product. This device is contraindicated for unstable fractures of the lower leg, ankle and foot.

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**APPLICATION INFORMATION:**

A cotton or cast sock will enhance comfort – **NOT INCLUDED**

**1) PREPARE BRACE**

- Unfasten straps and open liner.

**2) APPLY BRACE**

- While seated, place foot in brace (Fig. A).
- Position heel in back of brace. Wrap liner around foot and then ankle (Fig. B).
- Secure straps from bottom to top (Fig. C).

**3) REMOVE BRACE**

- To remove the brace, unfasten the straps, open the liner and remove the boot.

**CLEANING INSTRUCTIONS:**

The foam liner may be hand-washed in 86°F/30°C water with mild detergent and air-dried. Do not put in dryer or use other heat source to dry.

**MATERIAL COMPONENTS:**

Straps: WOVEN NYLON  
Liner: NYLON/POLYURETHANE/POLYESTER (OUTER/FOAM/INNER MESH)  
Binding: POLYESTER  
Outer Sole / Insole: EVA / RUBBER

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY AND DURABILITY, NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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