ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS:

The AirCast A60™ Ankle Support is designed to provide support and protection for the ankles. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction. Providing mild protection of a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS:

N/A

WARNINGS AND PRECAUTIONS:

- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or package has been opened.

Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

- Put on an absorbent sock**
 - Unfasten strap from black plastic ring, open top flaps, and open brace.
 - Slide foot, toe first, into top of brace (Fig. 1)
 - Pull brace stabilizers upward and align stabilizers with sides of ankle
- Wrap flaps around top of ankle and secure (Fig. 2)**
- Insert end of strap into black plastic ring**
 - Bend knee and align over toes
 - Pull strap until snug and secure strap end onto body of strap (Fig. 3)

CLEANING INSTRUCTIONS:

The entire A60 Ankle Support can be hand washed in lukewarm water with mild soap and air dried.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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