ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INDICATIONS

The Donjoy PSI Plus Shoulder Immobilizer is designed to support and immobilize the shoulder following injury or surgery.

PERFORMANCE CHARACTERISTICS:

Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction. Providing immobilization or controlled movement of a limb or body segment.

APPLICATION INSTRUCTIONS

1. Wrap and secure the waistband around the rib cage.
2. Shoulder strap: Bring strap over uninvolved shoulder and secure.
3. Wrap and secure the upper arm strap.
4. Wrap and secure the wrist strap.

CONTRAINDICATIONS

Do not use if you are allergic to any of the materials contained within this product.

WARNING & PRECAUTIONS

This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should only be used after consulting a health care professional if you suffer from skin diseases, injuries, lymphatic disorders, indistinct soft tissue swelling, or circulatory or sensory disorders in the affected area. The support should be snug but not impair circulation. Do not use over open wounds. If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues. Do not use the device if it is damaged and/or the packaging has been opened. NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

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CARE

Hand wash in warm water with mild detergent. Rinse thoroughly and air dry.

COMPOSITION

Polyester, Polypropylene, Polyamide

CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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VERWENDETE MATERIALIEN

Polyester, Polypropylene, Polyamid

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