ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

Rhizarthrosis

PERFORMANCE CHARACTERISTICS

- Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.
- Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS

- None

WARNINGS AND PRECAUTIONS

- We recommend that the first application be made under supervision of a health care professional.
- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

- 1) Open/release all straps
- 2) Insert hand into wrist strap and place brace around thumb
- 3) Fasten wrist strap: do not over-tighten (used to keep brace in correct position).
- 4) Fasten the thumb brace
- 5) Apply pressure on brace to fit thumb comfortably. Re-adjust, if required, to avoid pressure areas.

CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION

Aluminium, Polyethersulfone, Polyurethan, Polyamide, crosslinked polyolefin, PVC

 **NOT MADE WITH NATURAL RUBBER LATEX.**

FOR SINGLE PATIENT USE ONLY

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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