## ENGLISH

**BEFORE USING THE DEVICE. PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### Intended User Profile:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

### INTENDED USE/INDICATIONS

- Clavicular Fractures
- Postural Problems

### PERFORMANCE CHARACTERISTICS

Soft/good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.

### CONTRAINDICATIONS

- None

### WARNINGS AND PRECAUTIONS

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INSTRUCTIONS

Place triangular closure at the center of the back with the straps lying over the shoulders and to the front of the body.

Wrap the straps under the arm and secure at the triangular closure. Adjust the straps to the desired position.

### COMPOSITION

Polyamide, Cotton, Polyester, Polypropylene, Polyurethane

### CARE

Hand wash in cold water with mild detergent and air dry.

**NOT MADE WITH NATURAL RUBBER LATEX.**

**FOR SINGLE PATIENT USE ONLY**

**WARRANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOTICE:** WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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