DONJOY®

ISOFORM™ SIO

| | |
|-----------------|----|
| ENGLISH | 2 |
| ESPAÑOL | 4 |
| DEUTSCH | 6 |
| ITALIANO..... | 8 |
| FRANÇAIS..... | 10 |
| NEDERLANDS..... | 12 |
| PORTUGUÊS | 14 |
| ČEŠTINA | 16 |
| SLOVENČINA..... | 18 |
| DANSK..... | 20 |
| SVENSKA | 22 |
| SUOMI | 24 |
| NORSK..... | 26 |
| 简体中文 | 28 |
| 日本語 | 30 |

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




DJO, LLC
5919 Sea Otter Place, Suite 200
Carlsbad, CA 92010 • USA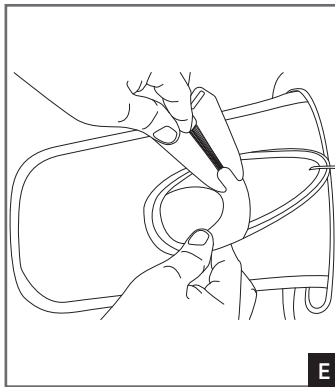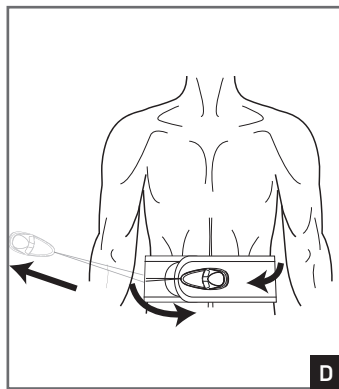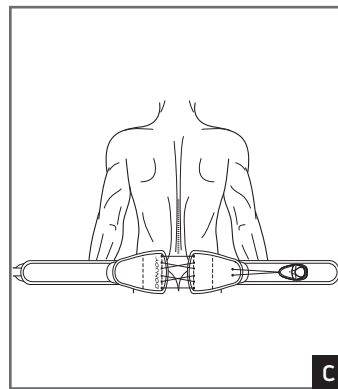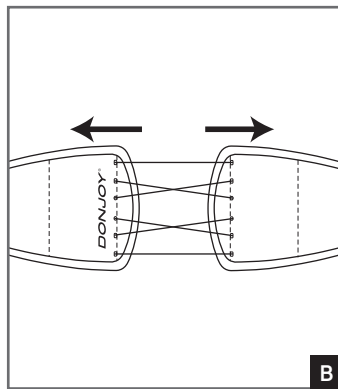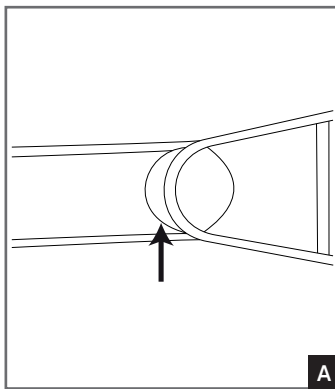

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a suitably trained professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: Designed to support the lower back and may be suitable for people with sacroiliac pain, dysfunction, lower back pain, sprains/strains.

PERFORMANCE CHARACTERISTICS: Soft-good/semi-rigid good designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: Pulmonary, cardiovascular or skeletal conditions which have risk to be made worse as a result of compression and/or pressure, unstable, displaced fractures. Do not use if you are allergic to any of the materials contained within this product.

WARNINGS AND CAUTIONS: If you experience any pain, swelling, sensation changes, excessive discoloration, irritation to the skin or any unusual reaction while using this product, consult your medical professional. If any distortion of the support panels, fraying or deterioration of lace cables is noticed, discontinue use. Brace is intended to be worn over clothing. Brace should be appropriately sized to the patient to ensure proper support and function. If the brace does not fit properly, it may be adjusted by a healthcare professional. Care should be taken with patients, especially the elderly, not to apply the brace too tightly, which MAY cause excessive pressure. Patients should be instructed on appropriate tensioning during ongoing use. Patient should see a healthcare professional for a replacement if a product quality issue occurs within the warranty timeframe. Brace should not be exposed to moisturizers, chemicals, or solvents, as they may affect durability. This product must be prescribed and fitted by a suitably trained professional. The frequency and duration of use should be determined by your healthcare professional. Do not modify the device without a healthcare professional present. Do not use device other than instructed in this IFU. Keep out of reach of children under three years of age to avoid accidental ingestion of the small parts. Do not use over open wounds. Do not use the device if it is damaged and/or the packaging has been opened. **NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

SIZING:

1. Begin by taking a measurement of the circumference of the patient approximately 1" (2.5cm) above the widest part of the hips. For patients whose circumferential measurement is between sizes, select the larger brace.
2. Match the patient's size to the Sizing Chart below.

| SIZE | CIRCUMFERENTIAL MEASUREMENT |
|------|-----------------------------|
| S/M | Less than 48" (122 cm) |
| L/XL | 49" - 60" (125 cm - 152 cm) |

FITTING:

3. To adjust waist belt circumference, align the tip of the hook and loop attachment on the rear panel section. Repeat for opposite side. (Figure A)

NOTE: The pull tab may be oriented on either the right or left-hand side of the patient, however the loop exterior with the donning mitt should be secured on the opposite side of the pull tab.

APPLICATION INSTRUCTIONS:

4. Loosen the brace by detaching the pull tab from the loop exterior and pulling the brace in opposite directions. (Figure B)
5. Wrap the belt approximately 1" (2.5cm) above the widest part of the hips, centering the rear section at the midline of the lower back so that the belt is just above the crease of the buttocks. Secure hook portion of donning mitt to the loop exterior. (Figure C)

NOTE: When reapplying the brace, it is easiest to guide the belt by inserting a hand

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in the donning mitt pocket.

6. Before tightening the brace, make sure the rear section is positioned evenly on both sides of the patient.
7. To tighten the brace, slip your thumb through the hole in the pull tab. Pull away from your body. When tightened, secure the pull tab to the loop exterior. Adjust compression as needed. (Figure D)

BRACE REMOVAL

8. Detach the pull tab from the loop exterior to loosen the brace. Unhook donning mitt from the loop exterior and remove brace.

ADJUSTING PULL TAB LENGTH (OPTIONAL)

9. Remove lace spool from the pull tab pocket and wind lace around the spool to shorten the length of the pull tab. Unwind lace from the spool to lengthen the pull tab. (Figure E)
10. Insert lace spool back into pull tab pocket.

CLEANING INSTRUCTIONS:

Hand wash in water (86°F/30°C), using soap. Air dry.

NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

Do NOT iron.

Do NOT tumble dry.

Do NOT bleach.

MATERIAL CONTENTS: Nylon, Polyester, Polyurethane, Polyethylene, Cotton, Acrylic, Polypropylene, Silicone, Synthetic Rubber, EVA.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

INTENDED FOR SINGLE PATIENT USE.

NOT MADE WITH NATURAL RUBBER LATEX.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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