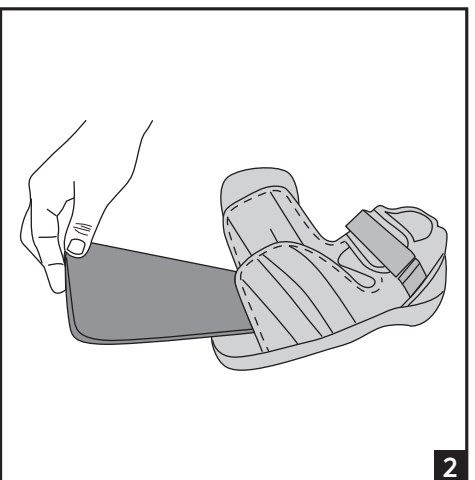
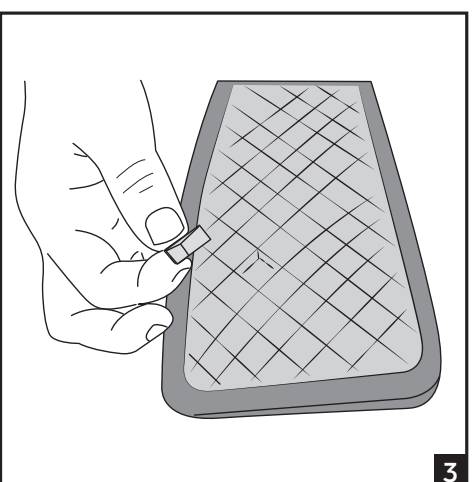
1

2

3

4

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### INTENDED USE/INDICATIONS:

The ProCare Squared Toe Post-Op Shoe Impax Grid Insole is a replacement insole designed to provide protection to the foot. Providing mild protection of a limb or body segment for acute and prophylactic care.

**CONTRAINDICATIONS:** Do not use if you are allergic to any of the materials contained in this product.

### APPLICATION INFORMATION

- 1) Open forefoot flaps of the Squared Toe Post-Op Shoe (Fig. 1)
- 2) Firmly grasp the edge of the gray Squared Toe Post-Op Shoe insole and remove from shoe. (Fig. 2)
- 3) With the Impax Grid Insole in hand, locate and remove the pre-cut foam cubes from the bottom of the insole as desired to relieve pressure. (Fig. 3)
- 4) Place the Impax Grid Insole (pink side down) into the shoe. The blue side of Impax Grid Insole should be against bottom of foot. (Fig. 4)

### WARNINGS AND PRECAUTIONS:

It is recommended to have the device fitted by a suitably qualified healthcare professional. Do not use over open wounds. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. Do not use this device if it was damaged and/or packaging has been opened. Discontinue use if there are markings, blistering or discoloration of the skin, particularly over the heels and bony prominences. The use of compression in patients with an ABPI less than 0.8 should only be initiated under specialist advice and requires very close monitoring and review. Use with caution in patients with diabetes, who may have unreliable ABPIs due to artificial calcification as well as an underlying sensory neuropathy.  
**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

### MATERIAL COMPOSITION:

Insole: Ethylene Vinyl Acetate

### FOR SINGLE PATIENT USE ONLY

~~NOT MADE WITH NATURAL RUBBER LATEX.~~

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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