## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

### INTENDED USE/INDICATIONS

All indications requiring immobilisation of the thumb joints - CMC/TMC joint (Carpometacarpal / Trapeziometacarpal) and / or MCP joint (Metacarpophalangeal) - such as: sprains, subluxations, dislocations, rhizarthrosis, ligament injuries.

### PERFORMANCE CHARACTERISTICS

Providing immobilization or controlled movement of the limb or body segment.

### CONTRAINDICATIONS

- None

### WARNINGS AND PRECAUTIONS

- We recommend that the first application be made under supervision of a health care professional.
- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INSTRUCTIONS

- 1) Loosen straps and slip hand into brace
- 2) Place the thumb in the corresponding opening at the top of brace
- 3) Wrap and secure the wrist wrap.
- 4) Wrap and secure thumb strap.

Stay is malleable and may be contoured for individual patient needs. Adjust the tightness of the straps according to compression and comfort needed. Ensure that the edge of the brace with the buckles overlaps the opposite one.

### CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

### COMPOSITION

Aluminium, Polyester, Polyamid, Polyurethane, PVC, Cotton, Polypropylene

NOT MADE WITH NATURAL RUBBER LATEX.

### FOR SINGLE PATIENT USE ONLY

**WARRANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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