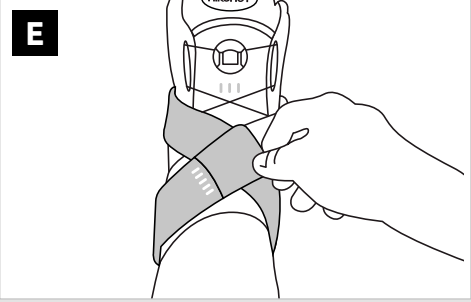
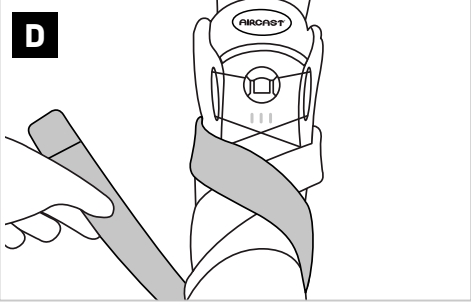
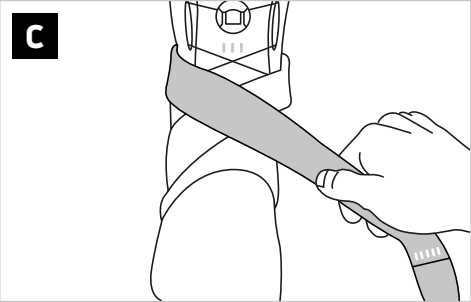
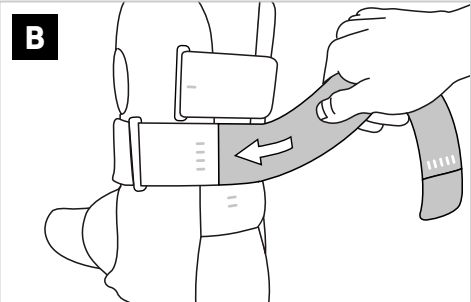
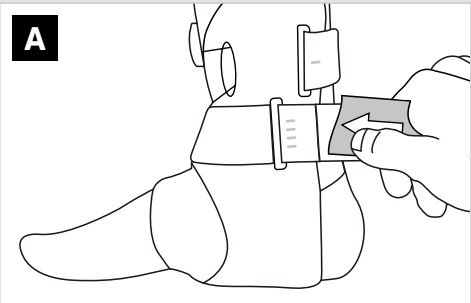
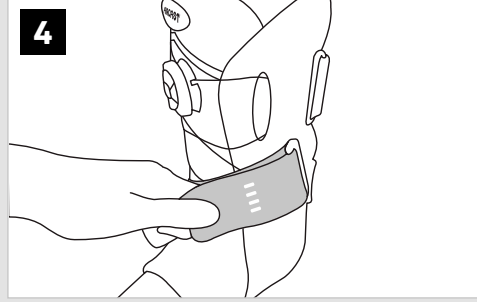
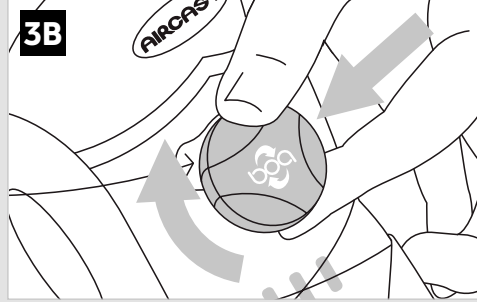
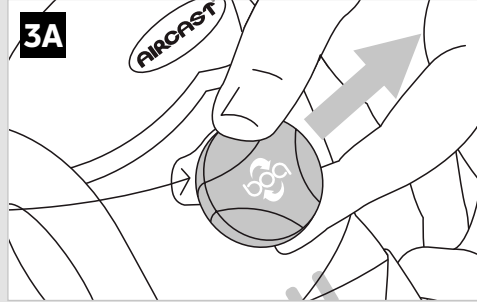
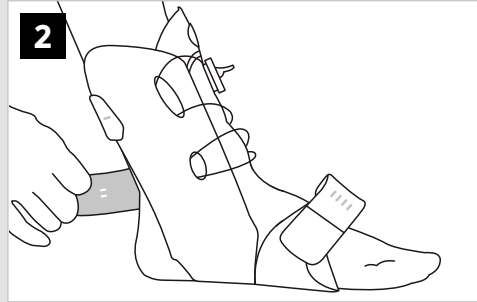
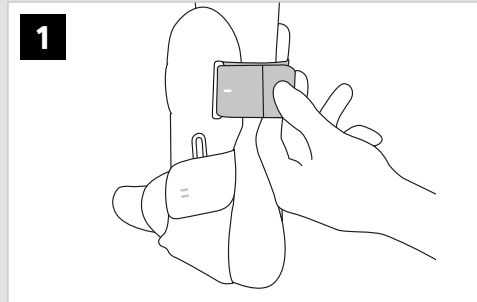
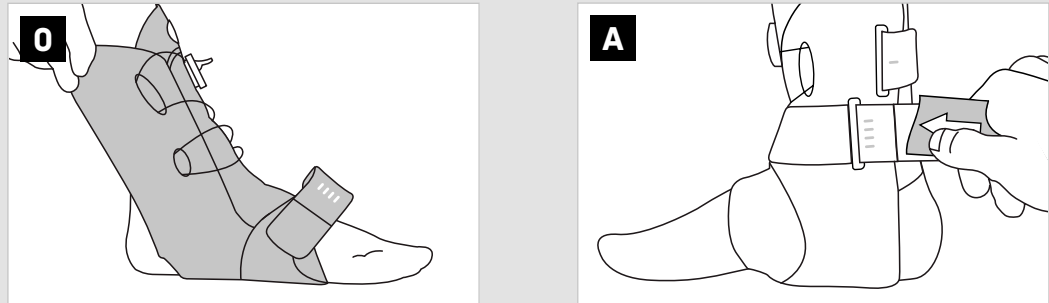
DEUTSCH

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ENGLISH

Before using the device, please read the following instructions completely and carefully. Correct application is vital to the proper functioning of the device.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to:

- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

This Aircast Airgo Plus is designed to provide support, and stabilize the ankle following injuries such as ankle ligament tears, ankle sprains, and ankle instabilities. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

KONTRAINDICATIONS

Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS

The brace should only be worn under supervision of a health care professional. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately. If suffering from the following conditions, the ankle brace should only be used with the consent of a health care professional:

- Lymphatic drainage disorders
- Circulatory disorders
- Sensory disorders
- Skin diseases or skin irritations
- Open wounds
- Lymphedema, swelling

Do not use over open wounds. Do not use this device if it was damaged and/or package has been opened.

Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device. For maximum comfort and support wear an absorbent sock and a laced athletic shoe when wearing the brace.

BOA® LACING SYSTEM (FIG. 3)

- To Loosen BOA® (Fig. 3a):
 - Lift BOA Knob.
 - Pull the lacing
- To Close and Tighten BOA® (Fig. 3b):
 - Push BOA Knob down and then turn BOA Knob clockwise.
 - BOA Knob must be in the down locked position when wearing brace.

APPLICATION INSTRUCTIONS

- Loosen the laces, unfasten straps and open brace.
 - Slide foot into the back of brace. Pull the lateral shells upwards and align with the sides of the ankle (Fig. 0).
 - Close the straps according to numbering:
- Pull the top strap around the back of the calf and fix to the external side of the brace without tightening too much (Fig. 1). This helps position the lateral shells correctly.
 - Pull the bottom Achilles strap around the back of ankle and fix to the external side of the brace without tightening too much (Fig. 2).
 - Tighten the brace by turning the BOA Lacing System clockwise until the desired level of support (Fig. 3). Note: If the ankle brace is too tight, open the BOA system, loosen the laces, push the knob down and re-tighten.
 - Pull the front cross-strap tightly through the buckle and fix firmly onto front of brace (Fig. 4).

FIGURE-8 STRAP

- To increase stability, you can extend the front cross-strap by using the figure-8 strap. Attach it to the cross-strap. The arrow on the strap should be visible.
- Wrap the extended strap, centered around your Achilles tendon.
- Bring the strap forward, crossing over your forefoot.
- Run the strap under your foot.
- Fasten the strap across the front.
- If the figure-8 strap is too long, shorten it.

CARE

- Hand wash in lukewarm water (30°C) with a mild detergent
- Air dry (do not place on radiators/heaters or other heat sources)
- Do not use bleach or chlorine, do not iron.

COMPOSITION

Plastic, Silicone, PES (polyether sulfone), Polyester, Polyamide, Elastane, natural rubber (latex), Polyurethane, Nylon

CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS. (< 0.1% - NOT IN DIRECT CONTACT WITH SKIN)

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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©2025 DJO, LLC