ENGLISH

Before using the device, please read the following instructions completely and carefully. Correct application is vital to the proper functioning of the device.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user or caregiver should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use

INTENDED USE/INDICATIONS

The DonJoy Lombothrap is designed to provide comfort and support of the lumbar spine. It may be suitable for the symptomatic relief of acute and chronic low back pain. Performance Characteristics: Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS

If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional. Do not use if you are allergic to any of the materials contained within this product.

WARNINGS AND PRECAUTIONS:

- For maximum efficacy, ensure the correct size of strap is selected and the side pulls are adjusted to the required fit.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of the device.

APPLICATION INSTRUCTIONS

- Position the back support by placing the dorsal part against the lower back and taking care to center it along the axis of the spine. (Fig. 1)
- Adjust and tighten the back support around the waist according to the support required (Fig. 2). Secure using the contact closure at the front of the support. (Fig. 2a)
- Adjust and tighten the lateral straps, taking care to center their central part along the axis of the spine. (Fig. 3)
- Put on the edges of both lateral straps, at the same time, and secure them on the front. (Fig. 4)

CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

DISPOSAL

Dispose of the device and its packaging in accordance with local or national environmental regulations.

COMPOSITION

Polyamide, Polyester, Polyurethane, Elastane, Steel, Polypropylene

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.

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