NOT MADE WITH NATURAL RUBBER LATEX

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DONJOY®

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to:

- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

• Treatment of acute or chronic low back pain, lumbar arthrosis.

CONTRAINdications

• None

WARNING & PRECAUTIONS

The product is intended for single patient use only. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

APPLICATION INSTRUCTIONS

For maximum efficiency, make sure to choose the proper size and adjust the straps to the required compression.

- 1) Open the 4 straps.
- 2) Position the support around the waist and close using contact closure.
- 3) Pull the straps to the desired level of compression and affix them on the front of the support, using the contact closure tabs.

CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION

Polyamide, Polyester, PVC, Rubber, Elasthane, Steel

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©2023 DJO, LLC