ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER

The intended user should be the patient, a licensed healthcare professional, or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS

The DonJoy LumboForce® 4 was developed to provide comfort and support for the lumbar spine. Bridging orthosis to relieve the lumbar spine. A versatile (removable) disc, stage 3: extension & stage 4: sequestration

- Spondylolisthesis
- Stable vertebral fractures of the lumbar tract
- Lumbar spinal canal stenosis
- Spinal canal decompression

CONTRAINDICATIONS

If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional. Do not use if you are allergic to any of the materials contained within this product.

PERFORMANCE CHARACTERISTICS

- Providing support and stabilization to the limb or body segment.
- Providing partial immobilization or controlled movement to the limb or body segment.
- Providing mild protection of a limb or body segment for acute and prophylactic care.

WARNING AND PRECAUTIONS

We recommend that the first application be made under supervision of a healthcare professional. If you have skin conditions/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a healthcare professional. If you experience any discomfort, pain, swelling or sensation changes, or any unusual reactions while using this product, consult your healthcare professional immediately. This product is for single patient multiple use only. Do not use this device if it is damaged and/or packaging has been opened. The product should be removed during long periods of rest. Do not use during showering or bathing. Keep the product away from children. Do not use on open wounds and/or sensitive skin. Do not use this product if it is damaged or defective. Do not use this product in any way that may damage its integrity or use the device other than intended. The frequency and duration of use should be determined by a healthcare professional. **NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to use of this device.**

APPLICATION INSTRUCTIONS

1. Open the lumbar belt and loosen all straps.
2. Before placement, your healthcare professional should adjust the rigid aluminum back frame by adjusting the upper and lower ends of the frame, lengthening them if necessary horizontally using the clips according to the width of your back and adapting the aluminum frame by bending it by hand, according to the shape of your body.
3. The rigid back insert of the aluminum frame is fastened by hook-and-loop in the center on the inside of the back support.
4. Place the lumbar support around the waist. Make sure the lumbar belt is correctly centered on your back and the lumbar support is centered on your lumbar spine.
5. Hold the back support by inserting your hands into the comfortable pockets, pull to expand the main fabric and wrap it around your waist. The front closure panels should be closed parallel to each other if you are a man and at a slight angle if you are a woman. Then remove your left hand from the grip pockets and, with your right hand, press down firmly on the top front closure panel. The lateral pullers are pulled forward equally with gentle force until the desired degree of tension is achieved. They are then fastened on the front closure panels, with the hook-and-loop. The pullers can be shortened or lengthened by wrapping/unwrapping the cord around the pulley tab.
6. Pull both lateral straps forward equally using light pressure until the desired level of support is achieved. Then attach them to the front closure panels using the hook-and-loop fastener. The length of the straps can be customized by cutting them.
8. The back support should be fastened tightly. Fit your body well, and provide comfort and relief (the back support should not apply any pressure to the groin when sitting down). A tightening readjustment may be necessary to adapt belt compression and to avoid pressure points on the groin when sitting down.

CLEANING INSTRUCTIONS

Machine washable on gentle cycle (max 30°C) with a mild detergent. Hook and loop of the straps should be closed for washing to avoid any damage. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not iron.

COMPOSITION

Front Closure : Polyamide, Acrylic Resin, Polyurethane, Polyester, Polyolefin and Polyoxymethylene. Belt : Polyamid, Elastane, Polyester, Steel, Polyurethane. Frame: Aluminum, Polyester, Polyamide, Polyamide with 30% glass fiber, Ethyl Cyanoacrylate, Polyurethane, Polypropylene and Synthetic Rubber.

WARRANTY D.J.O. LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, OF RESISTANCE, OF DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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