ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER

The DonJoy PSI Plus Shoulder Immobilizer is designed to support and immobilize the shoulder following injury or surgery.

INTENDED USE/INDICATIONS

The DonJoy PSI Plus Shoulder Immobilizer is designed to support and immobilize the shoulder following injury or surgery.

Performance characteristics:

- Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.
- Providing immobilization or controlled movement of a limb or body segment.

CONTRAINDICATIONS

Any known allergy to the materials of the medical device.

WARNING AND PRECAUTIONS

- This product is to be used under the supervision of a medical professional.
- This device is not intended for use by the general public without supervision.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- The support should only be used after consulting a health care professional if you suffer from skin diseases, injuries, lymphatic disorders, indistinct soft tissue swelling, or circulatory or sensory disorders in the affected area.
- The support should be snug but not impair circulation.
- Do not use over open wounds.
- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels, or similar residues.
- Do not use the device if it is damaged and/or the packaging has been opened.

NOTE: Contact the manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

Before using the product for the first time, it must be adjusted to the patient's measurements and the side of the affected shoulder.

- 1) Place the wide strap under the chest and fasten it. Note: Make sure that the closure with the hook-and-loop strip fastens in the direction of the affected shoulder.
- 2) Attach the upper arm strap on the side of the support and secure the upper arm.
- 3) Fasten the forearm strap in the center of the abdominal belt to support the wrist.
- 4) Place the shoulder strap over the uninjured shoulder and connect it to the hand strap. Adjust the length of the shoulder strap to relieve the injured arm.

For reuse, the upper arm strap and the forearm strap can remain attached to the belt.

CLEANING INSTRUCTIONS

Wash by hand with a mild detergent in warm water (30°C). Rinse thoroughly and air dry.

COMPOSITION

Polyester, Polypropylene, Polyamide, Acrylic, Pom (Polyoxymethylene), Polyurethane

NOT MADE WITH NATURAL RUBBER LATEX

FOR SINGLE PATIENT USE ONLY.

WARRANTY DJO, LLC will repair or replace all, or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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