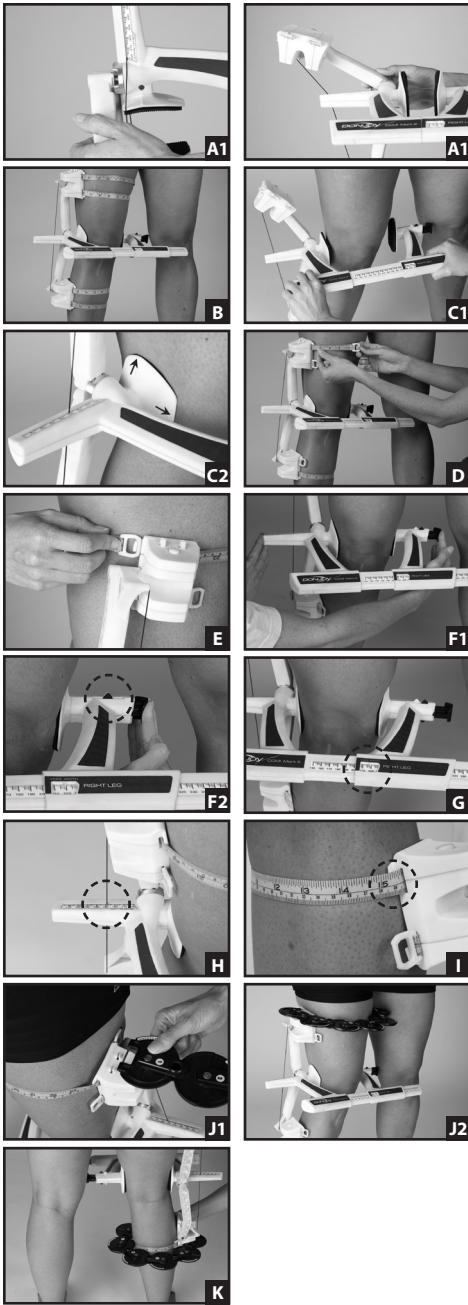
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS: To determine leg contour measurements for the construction of a DonJoy Custom Brace. For further instructions, please refer to the CCMI MARK III instruction DVD included with the instrument.

CONTRAINDICATIONS: None

WARNINGS AND PRECAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

APPLICATION INFORMATION:

- To unstow, distract and rotate swing arms as shown in figures (A1 & A2).
- Have the patient stand feet shoulder width apart and knees flexed slightly. Do not allow the knees to flex beyond 15 degrees. Apply the instrument directly to the bare leg. Note: Taking measurements from the uninvolved leg is not recommended because of differences in leg dimensions due to dominance of one leg over the other. See figure (B).
- Distract knee width caliber and position on the patient's knee. Arrows on condyle shells should align with the top of the knee cap and slightly posterior to midline. See figures (C1 and C2).
- For standard thigh and calf, extend tape measure marked in inches located distal to knee. Wrap around leg and secure to hook on tape module. Press tape release button to pull tape snug to leg; ensure it is parallel to floor. For short thigh or calf, follow same procedure using Metric measurements. See figure (D and E).
- To establish proper knee compression for knee width measurement, support lateral side of instrument and press load plunger on medial condyle until triangles are aligned, as shown in figures (F1 and F2).
- Record knee width on order form. See figure (G.)
- Record lateral offset on order form. See figure (H).
- Record thigh and calf circumference on order form. See figure (I).
- To record thigh and calf contour measurements, insert steel shaft of contour gauge into snap feature on tape module, as shown in figure (J1). Wrap the contour gauge around the anterior thigh, parallel with the floor being careful not to compress tissue. See figure (J2). Record the readings in each dial window starting with dial #1. NOTE: Dials #4 & 5 may be pulled away from the leg once a contour has been registered. This will make it easier to read. Remove contour gauge. Repeat the procedure at the posterior calf. See figure (K).

LIMITS OF ACCURACY: Tolerance for knee width and lateral offset measurement ± 1 mm (± 0.04 in). Tolerance for circumference measurements as applied to the leg of a normal patient: ± 2 mm (± 0.08 in).

USE/CARE/DISINFECTION

- To clean device and measuring tape, wipe with a damp cloth containing a mild soap. Remove soap with damp cloth and follow with a dry cloth.
- Between each use, disinfect condyle pads and measuring tape with alcohol wipes.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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