3

ENGLISH


Before using the device, please read the following instructions completely and carefully. Correct application is vital to the proper functioning of the device.

INTENDED USER

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

The DONJOY® NEXTEP XCEL™ Air Ankle Walker is designed to provide support and immobilization to the foot and ankle. It may be suitable for the treatment of severe acute ankle sprains, soft tissue injuries of the foot and ankle, stress fractures of the ankle, stable fractures of the foot, and ligamentous injuries such as plantar fasciitis

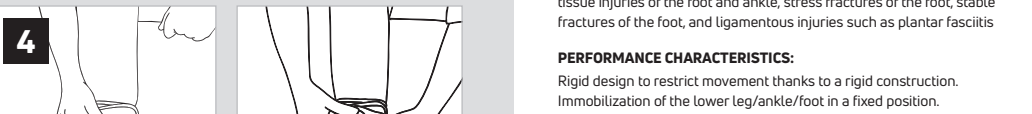PERFORMANCE CHARACTERISTICS:
Rigid design to restrict movement thanks to a rigid construction. Immobilization of the lower leg/ankle/foot in a fixed position.

CONTRAINDICATIONS

Do not use if you are allergic to any of the materials contained within this product. This device is contraindicated for unstable fractures of the lower leg, ankle and foot.

The determination of when to apply the brace rests strictly at the discretion of the treating physician.

WARNINGS AND PRECAUTIONS

- The product is intended for single patient use only.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- Do not overinflate aircells. Improper aircell inflation may cause severe skin irritation in patients with diabetic neuropathy. Reduce aircell compression with any visual skin changes or reported discomfort. Do not use this device on patients incapable of communicating physical discomfort.
- Do not use this device if you cannot feel sensations whether due to post-op anaesthesia, nerve, skin or other conditions.
- Do not use during showering or bathing.
- Do not use on open wounds(s) and/or sensitive skin.
- Do not wear or leave while wearing the brace as your ability to brake will be impaired.
- Use caution when walking on slippery or wet surfaces to avoid injury.
- At high altitudes (especially during flights) the aircell will expand beyond optimal level. In this case adjust aircell pressure.
- Always consult with your physician or therapist before making changes to the brace. This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised medical use.
- Always inspect before use. Check for broken stitches or parts, torn cut or frayed material, or buckles, or hook and loop fasteners that do not hold securely. Do not use damaged product. Discontinue use when torn, worn, frayed in any way. Never alter or repair this product. This product is not intended to be used with any other product except Enovis approved accessories. Use a sneaker or street shoe with a ShoeLift™ or equivalent on uninjured foot while ambulating to compensate for leg length changes while wearing a fracture walking brace.

APPLICATION INSTRUCTIONS

- Sit down to apply the DONJOY® NEXTEP XCEL™ Air Ankle Walker.
- Open the straps, remove the front panel of the walker.
- Insert your foot and ankle into the Walker and close the liner with the hook strips, make sure the toe piece is also folded on top of the forefoot.
- Place the front shell back into position and fasten the straps from the foot upward.

- To ensure it fits properly into the hole at the top of the front shell, the pump can be repositioned on the liner using the hook-and-loop attachment. On the upper strap tighten left and right side at the same time to keep the front panel centered. Do not over tighten the straps.
- As swelling patterns change it may be necessary to readjust the straps.
- To inflate air cells press and release the red air bulb until the aircell provides comfortable cushioning and support. Do not over inflate.
- To deflate or adjust compression, turn the deflate button and deflate until you are comfortable.
- To remove, deflate air cells and unfasten all straps without removing them completely from the assembled walker. Remove the front shell, unfasten the soft good liner and gently bring the foot and ankle forward and out of the walker.

CLEANING INSTRUCTIONS

Hand wash liners (at max. 30° C) using mild soap, rinse thoroughly. Air dry. Do not use bleach thoroughly, residual soap may cause irritation and deteriorate material. Plastic parts can be wiped with a cloth and mild soap or disinfectant. Then allow to dry before putting on the walker.

DISPOSAL
Dispose of the device and its packaging in accordance with local or national environmental regulations.

COMPOSITION

Polipropilene, PU foam laminate, Polyester, TPU, Nylon, TPR, EVA

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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