1

**ENGLISH**  
BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

**INTENDED USER PROFILE**

- The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INTENDED USE/INDICATIONS**

- Wrist sprains, strains and injuries.
- Pre/post-surgery wrist immobilization.

**PERFORMANCE CHARACTERISTICS**

Providing immobilization or controlled movement of the limb or body segment.

**CONTRAINDICATIONS**

- None

**WARNINGS AND PRECAUTIONS**

- We recommend that the first application be made under the supervision of a healthcare professional.
  - The support should only be used after consulting a health care professional if you suffer from skin diseases, injuries, lymphatic disorders, indistinct soft tissue swelling, or circulatory or sensory disorders in the affected area.
  - The support should be snug but not impair circulation.
  - If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should stop using the device and contact your healthcare professional immediately.
  - Do not use over open wounds.
  - If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.
  - Do not use the device if it is damaged and/or the packaging has been opened.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

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**APPLICATION INSTRUCTIONS**

- 1) Desserrez les sangles et glissez la main dans l'attelle.
  - 2) Place the thumb in the corresponding opening at the top of brace (left or right).
- Note:**
- a) La seconde ouverture doit rester vide.
  - b) The hand and wrist should be gripped between the two stays (palmar and dorsal).
  - c) If needed, move the dorsal stay to ensure correct position.
  - d) Stays are malleable and may be contoured to fit individual needs.
  - e) Tighten and secure the middle strap first, using the contact closure tab.
  - f) Then tighten and secure the two other contact-closure straps.

Adjust the tightness of the straps according to compression and comfort needed. Ensure that the edge of the brace with the buckles overlaps the opposite one.

**CARE**

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

**COMPOSITION**

Aluminum, Polyester, Polyamid, Polyurethane, PVC

NOT MADE WITH NATURAL RUBBER LATEX.

**FOR SINGLE PATIENT USE ONLY**

**GARANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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