ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

- Support and compression following mild wrist injury and mild wrist osteoarthritis
- Proprioceptive and neuromuscular stimulation through compression to support balance conditions and movements)
- Prevention of injury
- Mild wrist sprains
- Wrist tendinitis and mild tenosynovitis
- Wrist instability
- Carpal tunnel syndrome

Performance characteristics:

- Provide mild compression of limb or body segment through elastic construction.
- Provide mild protection of a limb or body segment for acute and prophylactic care.
- Soft-god/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS

- Any known allergies to the medical device materials.
- Peripheral arterial diseases.
- Impaired blood flow in the hand (e.g. with diabetes mellitus).
- Impaired lymphatic drainage or soft tissue swelling of unknown origin.

WARNING AND PRECAUTIONS

- The product is intended for single patient use only
- If you have skin conditions/trauma in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not apply on open wounds.
- The product is designed to be worn directly on the skin.
- Do not apply any lotions or ointments on the relevant part of the body while wearing the support.
- Remove the support during longer periods of rest.
- If patients experience pain, swelling, changes in sensation or other unusual reactions occur while using this product, they should contact their doctor immediately.
- Do not use this device if it is damaged and/or packaging has been opened.
- Do not use this product in any way that may damage its integrity.
- Handle with care: avoid fastening the straps' hook closure onto the knitted sleeve as it may cause damage to the appearance of the product when strap is removed.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

- Put the support over the wrist, placing the thumb in the opening on the side.
- Adjust the fit of the support as necessary.
- Ensure that the hole in the pad is placed over the bony prominence on the outer side of the wrist.
- For additional compression and stability, fix the hook of the removable strap to the loop piece of the support. Tighten the strap around the wrist according to your needs. The strap should be loosened or removed immediately in the event of tingling, discoloration, swelling, or numbness.

CLEANING INSTRUCTIONS

Machine washable at gentle cycle (max 40°C) with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not iron. Hook and loop should be closed for washing.

COMPOSITION

Elastan, Nylon, Polyester, Polyamide, Cotton, Polyurethane, Elastomer, Polyolefin, Steel, Silicone, Polypropylene

NOT MADE WITH NATURAL RUBBER LATEX

FOR SINGLE PATIENT USE ONLY.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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© 2024 DJO, LLC