Playmaker Xpert

SLEEVE

WRAP

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

The DonJoy Playmaker Xpert Sleeve is designed to support the knee following mild MCL and/or LCL sprains and mild arthritis. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS :

Do not use if you are allergic to any of the materials contained in this product.

WARNING & PRECAUTIONS

- This product must be prescribed and fitted by a healthcare professional.
- The frequency, duration of use and directions for use should be determined by your healthcare professional.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- When worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.
- Do not use over open wounds.
- Do not use this device if it is damaged and/or the packaging has been opened.
- The support should be snug but not impair circulation.
- Please be advised that DJO, LLC, does not recommend simultaneous wearing of functional knee braces bilaterally (right and left legs), during activities (ie skiing) that may cause the braces to contact each other, resulting in potential loss of control and injury. Functional knee braces are not recommended to be worn over clothing.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

Unfasten and loosen the straps at the thigh and calf.
1.

- If using Sleeve version, pull the brace onto the leg.
- If using Wrap version, simply wrap around leg and fasten over the front of the leg.

2. Position the brace so that the hinges are parallel to each other, and the top of the circular cut-out aligns with the top of the kneecap.

3. Fasten the support straps at the thigh and calf.

4. Removable polycentric hinges

CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

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NOT MADE WITH NATURAL RUBBER LATEX RX ONLY

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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