ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

• The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

All indications requiring immobilization of one or two fingers, such as: sprains, subluxations, dislocations, fractures, post-trauma, pre- or postoperative immobilization.

PERFORMANCE CHARACTERISTICS

Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS

• None

WARNINGS AND PRECAUTIONS

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- We recommend that the first application be made under supervision of a health care professional.
- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

- 1) Position splint on affected finger.
- 2) Secure finger tip strap.
- 3) Secure palm strap around the palm. Palm strap may be repositioned if necessary.
- 4) Secure strap around the wrist.

Strap is malleable and may be contoured for individual patient needs. All straps should be secured to a snug but comfortable fit.

CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION

Aluminium, Polyester, Polyamid, Polyurethane, PVC, Polypropylene

NOT MADE WITH NATURAL RUBBER LATEX

FOR SINGLE PATIENT USE ONLY

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale to the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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