ENGLISH

Before using the device, please read the following instructions completely and carefully. Correct application is vital to the proper functioning of the device.

INTENDED USER

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

The DONJOY® NEXTEP XCEL™ Ankle is designed to provide support and immobilization to the foot and ankle. It may be suitable for the treatment of severe acute ankle sprains, soft tissue injuries of the foot and ankle, stress fractures of the foot, stable fractures of the foot, and ligamentous injuries such as plantar fasciitis.

PERFORMANCE CHARACTERISTICS:

Rigid design to restrict motion of through rigid construction. Immobilisation of the lower leg/ankle/foot in a fixed position.

CONTRAINDICATIONS

Do not use if you are allergic to any of the materials contained within this product. This device is contraindicated for unstable fractures of the lower leg, ankle and foot.

The determination of when to apply the brace rests strictly at the discretion of the treating physician.

WARNINGS AND PRECAUTIONS

- The product is intended for single patient use only.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- Do not overinflate aircells. Improper aircell inflation may cause significant irritation in patients with diabetic neuropathy. Reduce aircell impression with any visual skin changes or reported discomfort. Do not use this device on patients incapable of communicating physical discomfort.
- Do not use this device if you cannot feel sensations whether due to post-op anaesthesia, nerve, skin or other conditions.
- Do not use during showering or bathing.
- Do not use on open wound(s) and/or sensitive skin.
- Do not drive while wearing the brace as your ability to brake will be impaired.
- Use caution when walking on slippery or wet surfaces to avoid injury.
- At high altitudes (especially during flights) the aircells will expand beyond optimal level. In this case adjust aircell pressure.
- Always consult with your physician or therapist before making changes to the brace. This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use.
- Always inspect before use. Check for broken stitches or parts; torn, cut or frayed material; or buckles, or hook and loop fasteners that do not hold securely. Do not use damaged product. Discontinue use when torn, frayed in any way.
- Never alter or repair this product. This product is not intended to be usedwith any other product except Envois approved accessories.
- Use a sneaker or street shoe with a ShoeLift™ or equivalent on uninjured foot while ambulating to compensate for leg length changes while wearing a fracture walking brace.

APPLICATION INSTRUCTIONS

- Sit down to apply the DONJOY® NEXTEP XCEL™ Ankle Walker.
- Open the straps, remove the front panel of the Walker.
- Insert your foot and ankle into the Walker and close the liner with the hook strips; make sure the toe piece is also folded on top of the forefoot.
- Place the front shell from anterior on the foot and leg and fasten the hook and loop straps beginning at the toe then moving up the limb. On the upper strap tighten left and right side at the same time to keep the front panel centered. Do not over tighten the straps. As swelling patterns change it may be necessary to readjust the straps.
- To remove, unfasten all straps without removing them completely from the assembled walker. Remove the front shell, unfasten the soft good liner and gently bring the foot and ankle forward and out of the walker.

CLEANING INSTRUCTIONS

Hand wash liners (at max. 30° C) using mild soap, rinse thoroughly. Air dry. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

Plastic parts can be wiped with a cloth and mild soap or disinfectant. Then allow to dry before putting on the walker.

DISPOSAL

Dispose of the device and its packaging in accordance with local or national environmental regulations.

COMPOSITION

Polipropilēne, PU foam laminate, Polyester, TPU, Nylon, TPR, EVA

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and accessories for material or workmanship defects for a period of 12 months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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