1

**ENGLISH**

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER

The intended user should be the patient, a licensed healthcare professional, or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the instructions, warnings and cautions in the information for use.

INDICATIONS

The LumboForce® Sacro is designed to stabilize and support the pelvis and lower back and to help relieve pain in the sacrum (Si Joint) and pelvis.

- Provide stabilization and pain relief following trauma to the symphysis/iliosacral joint
- Structural loosening of the sacro-iliac joint

CONTRAINDICATIONS

If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional. Do not use if you are allergic to any of the materials contained within this product.

PERFORMANCE CHARACTERISTICS

- Providing support and stabilization to the limb or body segment.
- Providing partial immobilization or controlled movement to the limb or body segment.
- Providing mild protection of a limb or body segment for acute and prophylactic care.

WARNING AND PRECAUTIONS

We recommend that the first application be made under supervision of a healthcare professional. If you have skin conditions/injuries on the relevant part of the body, the product should be fitted and worn only after consultation with a healthcare professional.

If you experience any change, pain, swelling or sensation changes, or any unusual reactions while using this device, consult your healthcare professional immediately.

This product is for single patient multiple use only and is not reusable.

Do not use this device if it was damaged and/or packaging has been opened.

The product should be removed during long periods of rest.

Do not use this product away from children.

Keep this product away from sensitive skin.

Do not use this device if it is damaged or defective.

Do not use this product in any way that may damage its integrity or use the device other than intended.

The frequency and duration of use should be determined by a healthcare professional.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

A short shirt or vest should be worn underneath the back support. If possible, the back support should be put on while standing.

1. Open the sacrum belt and loosen all straps.

2. The removable elastomer back pads are fastened by hook-and-loop in the centre of the back support (apply symbol pointed upwards), with the stimulating dots facing the back's curve. Adjust the pads individually for targeted pain relief.

3. Place the lumbar strap around the waist. Make sure that the lumbar belt is correctly centred over your spine.

4. Hold the back support by inserting your hands into the comfortable grip pockets, pull to extend the main fabric and wrap it around your waist. The front closure panels should be closely parallel to each other if you are a man. Then remove your left hand from the grip pocket and, with the right hand, press down firmly on the top of the front closure panel.

5. Fasten the lateral straps forward evenly with gentle force until the desired level of support is achieved.

6. The back support should be fastened tightly, fit your body well, and provide comfort and relief! (The back support should not apply any pressure to the groin when sitting down.) A tightening readjustment may be necessary to adapt belt compression and to avoid pressure points on the groin when sitting down.

CLEANING INSTRUCTIONS

Machine washable at gentle cycle (max. 30°C) with a mild detergent. Hook and loop of the straps should be closed for washing to avoid any damage. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not iron.

COMPOSITION

Cierre frontal: Poliamide, Acrylic Resin, Polyurethane, Polyester, Polyolefin & Polyoxymethylene.
Belt: Polymide, Elastane, Polyurethane, Polyester, Polypropylene, Polyoxymethylene & Steel.
Elastomer back pads: Polypropylene, Polypropylene 30 Fiberglass, Synthetic rubber & Styrene Block Copolymers.

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LATEX

NOT MADE WITH NATURAL RUBBER LATEX

GARANTY: DJO, LLC will repair or replace any part of the apparel and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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2

**APPLICATION INSTRUCTIONS**

A short shirt or vest should be worn underneath the back support. If possible, the back support should be put on while standing.

1. Open the sacrum belt and loosen all straps.

2. The removable elastomer back pads are fastened by hook-and-loop in the centre of the back support (apply symbol pointed upwards), with the stimulating dots facing the back's curve. Adjust the pads individually for targeted pain relief.

3. Place the lumbar strap around the waist. Make sure that the lumbar belt is correctly centred over your spine.

4. Hold the back support by inserting your hands into the comfortable grip pockets, pull to extend the main fabric and wrap it around your waist. The front closure panels should be closely parallel to each other if you are a man. Then remove your left hand from the grip pocket and, with the right hand, press down firmly on the top of the front closure panel.

5. Fasten the lateral straps forward evenly with gentle force until the desired level of support is achieved.

6. The back support should be fastened tightly, fit your body well, and provide comfort and relief! (The back support should not apply any pressure to the groin when sitting down.) A tightening readjustment may be necessary to adapt belt compression and to avoid pressure points on the groin when sitting down.

The frequency and duration of use should be determined by a healthcare professional.

NOTE: Contact the manufacturer and competent authority in case of a serious incident arising due to usage of this device.

RECOMMENDATION: Contact the manufacturer and competent authority in case of a serious incident arising due to usage of this device.

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