SVENSKA

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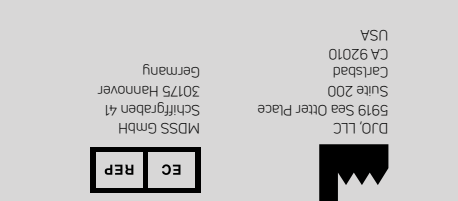
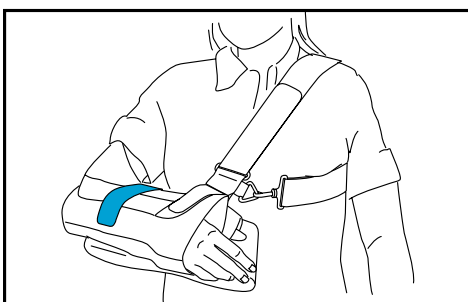
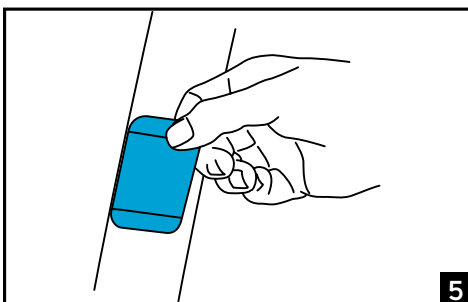
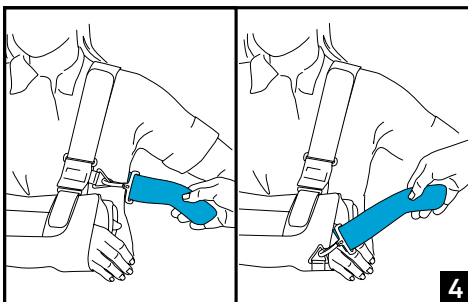
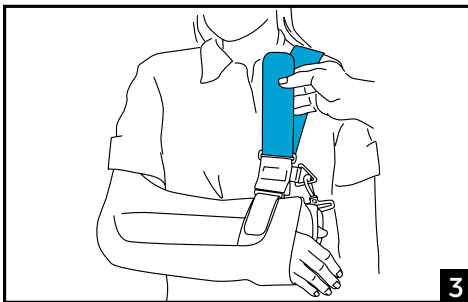
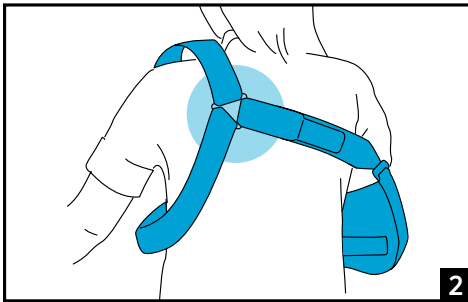
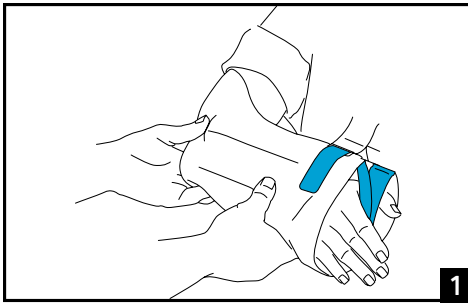
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: Provides arm support for injuries and a variety of pre- and post-op indications involving the shoulder, elbow, proximal or mid humerus, and wrist. These indications include fractures, dislocations, and soft tissue injuries to muscle, ligaments, or tendons.

PERFORMANCE CHARACTERISTICS:

- Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.
- Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS: N/A

WARNINGS AND PRECAUTIONS:

- If you have skin disorders/ injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of serious incident arising due to usage of this device.

APPLICATION INFORMATION:

Arm Immobilizer Sling

- Leave top strap on immobilizer/sling closed. Insert hand into sling with thumb on one side of hand guard and remaining fingers on the other side of guard.
- Correctly align plastic triangle that connects all three straps over scapula (shoulder blade) of uninjured arm. The length of the back strap may be adjusted by detaching the strap end and reattaching it along the strap length. Loosely drape shoulder strap over shoulder.
- Insert end of shoulder strap into top ring on Immobilizer and pull through. Snugly secure strap end to shoulder strap in front of torso. If necessary, adjust strap length (see step 2).
- Wrap underarm strap under healthy arm and insert strap end into Immobilizer clip. Secure strap end to strap snugly. **NOTE:** Clip may be secured to ring located on base of arm sling if necessary to prevent discomfort around breast area.
- Adjust straps for comfort and correct arm position as directed. If strap length is too long, reposition self adhesive tab on underside of strap. Straps may also be trimmed with scissors.

PERFORMING EXERCISES AS DIRECTED BY HEALTHCARE PROFESSIONAL:

- Open top strap on front of Immobilizer. Remove hand from hand guard and gently remove arm from Immobilizer — leave back strap, shoulder strap, and underarm strap attached to Immobilizer.

TO REMOVE ARM IMMOBILIZER

- Unclip underarm strap from immobilizer.
- Undo shoulder strap and remove immobilizer.

CLEANING INSTRUCTIONS:

Hand wash Arm Immobilizer Sling in cold water with mild soap and air dry.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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