## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### INTENDED USE/INDICATIONS

All indications requiring immobilisation of the wrist and thumb, such as: sprains, strains, trauma, tendinopathy, rhizarthrosis, carpal tunnel syndrome, post-trauma, pre- or postoperative immobilisation.

### PERFORMANCE CHARACTERISTICS

Providing immobilization or controlled movement of the limb or body segment.

### CONTRAINDICATIONS

- None

### WARNINGS AND PRECAUTIONS

- We recommend that the first application be made under supervision of a health care professional.
- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INSTRUCTIONS

- 1) Loosen straps and slip hand into brace
- 2) Place the thumb in the corresponding opening at the top of brace
- 3) Tighten and secure the middle strap first, using the contact closure tab.
- 4) Then tighten and secure the two other contact-closure straps.
- 5) Wrap and secure thumb strap.

Straps are malleable and may be contoured to fit individual needs. Adjust the tightness of the straps according to compression and comfort needed.

### CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

### COMPOSITION

Aluminium, Polyester, Polyamid, Polyurethane, PVC, Cotton, Polypropylene

**WARRANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**NOTICE:** WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY, FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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