## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER

The intended user should be a licensed medical professional, the patient, the patient’s caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### INTENDED USE/INDICATIONS

- Support and compression following acute and chronic knee injury and mild knee osteoarthritis
- Proprioceptive and neuromuscular stimulation through compression (to support balance conditions and movements)
- Patellofemoral pain (knee pain)
- Mild knee instability
- Prevention of injury

### Performance characteristics:

Provides compression (French Standard ASQUAL Class III: 20-36 mmHg and German Standard RAL Class II: 23-32 mmHg) of limb or body segment through elastic construction.

- Provide mild protection of a limb or body segment for acute and prophylactic care

### CONTRAINDICATIONS

- Any known allergies to the medical device materials.
- Peripheral arterial diseases.
- Impaired blood flow in the leg (e.g. with diabetes mellitus).
- Impaired lymphatic drainage or soft tissue swelling of unknown origin.

### WARNING AND PRECAUTIONS

- The product is intended for single patient use only
- If you have skin conditions/trauma in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not apply on open wounds.
- The product is designed to be worn directly on the skin.
- Do not apply any lotions or ointments on the relevant part of the body while wearing the support.
- Remove the support during longer periods of rest.
- If patients experience pain, swelling, changes in sensation or other unusual reactions occur while using this product, they should contact their doctor immediately.
- Do not use this device if it is damaged and/or packaging has been opened.
- Do not use this product in any way that may damage its integrity.

**NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.**

### APPLICATION INSTRUCTIONS


- Sit down to fit the Genuforce.
  - Grip edges of the support at the larger opening.
  - Pull the support over the knee.
  - Adjust the fit of the support as necessary.
- Make sure that the circular pad encloses the kneecap.

### CLEANING INSTRUCTIONS

Machine washable at gentle cycle (max 40°C) with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not iron.

### COMPOSITION

Elastane, Nylon, Steel, Polypropylene, Polyurethane, Elastomer, Silicone, Polyester, Polyolefin, Polyamide, Cotton

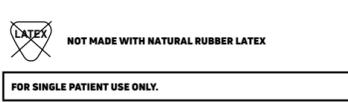**FOR SINGLE PATIENT USE ONLY.**

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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