ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

- Helps to relieve pain in case of ball problems at the little toe (little toe bunion/bunionette/talon bunion).
- Helps to protect against further pressure, friction and irritation.
- Helps to prevent corns and calluses, and sore skin and swelling.

Performance characteristics: Providing mild protection of a limb or body segment for acute and prophylactic (preventive) care.

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CONTRAINDICATIONS

- If you have the following conditions please consult a physician before using the device: gout attack, peripheral arterial disease or impaired blood circulation, rheumatic diseases, hypersensitivity to pressure, base joint arthritis, polyneuropathy or soft tissue swelling.
- Do not use this device if you have skin diseases or injuries in the affected area of the body.
- Any known allergies to the medical devices materials.

Performance characteristics: Providing mild protection of a limb or body segment for acute and prophylactic (preventive) care.

WARNING & PRECAUTIONS

The product is intended for single patient use only. The product must be worn directly on the skin. Do not apply any lotions or ointments on the relevant part of the body while wearing the support.

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact a medical professional immediately.

Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

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APPLICATION INSTRUCTIONS

Apply to clean and dry skin. Slip the little toe through the loop. Make sure to slide the loop all the way down to the base of the toe. The pad should be over the area of the bunionette.

Performance characteristics: Providing mild protection of a limb or body segment for acute and prophylactic (preventive) care.

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Cleaning instructions: Hand wash in warm water with mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not rub. Powder if necessary.

CLEANING INSTRUCTIONS

Cleaning instructions: Hand wash in warm water with mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not rub. Powder if necessary.

Performance characteristics: Providing mild protection of a limb or body segment for acute and prophylactic (preventive) care.

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