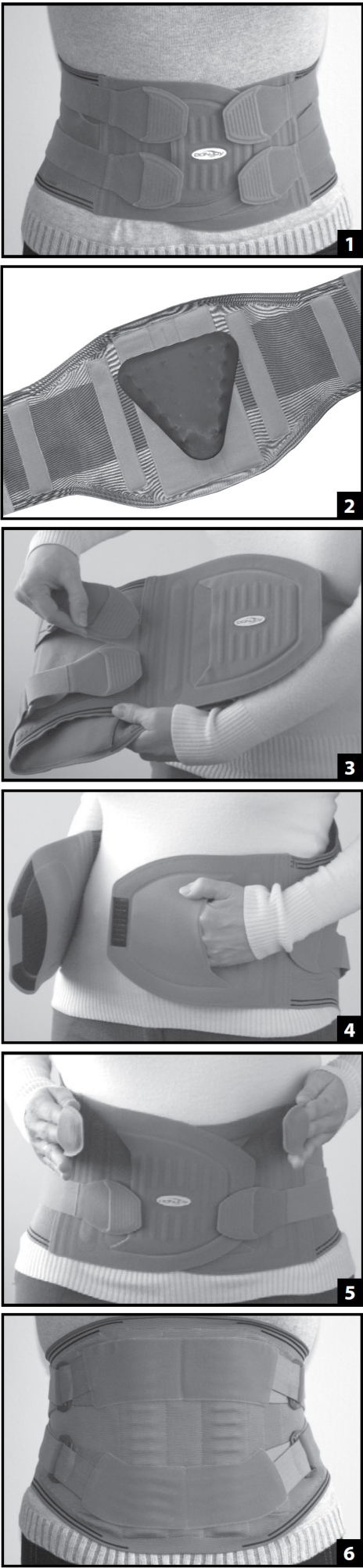
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver.

The user or caregiver should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS: The DonJoy LUMBOFORCE 3 is designed to provide comfort and support of the lumbar spine. It may be suitable for the symptomatic relief of acute and chronic low back pain.

Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.

Do not use if you are allergic to any of the materials contained within this product.

APPLYING THE BACK SUPPORT:

- If possible, the back support should be put on when standing. The back support is strapped on correctly if the blue edge is at the top and the lower edge covers the iliac crest. The lateral pull straps can be tightened according to individual support needs and then fastened onto the front closure panels, using the hook-and-loop. (Fig. 1)
- The moldable dorsal stays can be individually adjusted, without using tools, to the shape of the lumbar spine and the desired amount of support. If required, an optional removable silicone dorsal pad can be attached in the center on the inside of the back support, with the narrow end pointing downwards. The silicone dorsal pad is not included in the delivery and can be purchased as an optional accessory. (Fig. 2)
- Before applying the back support, the end edges of the lateral pull straps are fastened on the side, by hook-and-loop, so they are easy to reach. (Fig. 3)
- Hold the back support by inserting your hands into the comfortable grip pockets, and wrap it around your waist. When doing it, the (optional) silicone dorsal pad should be positioned in the center, over the lumbar spine. The front closure panels should be closed parallel to each other if you are a man, and at a slight angle if you are a woman. Then remove your left hand from the grip pocket and, with the right hand, press down firmly on the top front closure panel. (Fig. 4)
- The lateral pull straps are pulled forward equally with gentle force, until the desired of support level is achieved. They are then fastened on the front closure panel, with the hook-and-loop. (Fig. 5)
- The back support should be fastened tightly, fit your body well and provide comfortable support and relief. The back support should not apply any pressure to the groin when sitting down. (Fig. 6)

WARNINGS AND PRECAUTIONS:

- For maximum efficacy, ensure the correct size of strap is selected and the side pulls are adjusted to the required fit.
- If you experience any pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your medical professional immediately.
- Do not use over open wounds.
- Do not use the device if it is damaged and/or the packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

USAGE INSTRUCTIONS: The skin must have no injuries and should be free of moisture, oils, creams, gels or similar residues when putting on the device. A cotton shirt or vest should be worn underneath the back support. Please ensure the back support fits firmly and correctly.

CARE: The back support can be cleaned by hand-washing at 30°C. Air dry. Do not dry on a radiator. Do not iron. Do not bleach, chlorinate or clean with chemicals.

MATERIALS USED: Polyamide, polyester

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

In other circumstances, the currently applicable legal regulations apply.

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