## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:** The Procare ShoeLift is designed to compensate for leg length changes while wearing a fracture walking brace. Recommended for use on a securely fastened closed toe shoe or sneaker. Providing mild protection of a limb or body segment.

**CONTRAINDICATIONS:** None

### WARNINGS AND PRECAUTIONS:

- Do not use over open wounds.
- Do not modify the device or use the device other than intended.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it is damaged and/or the packaging has been opened.
- Always wear a securely fastened shoe.
- Remove the ShoeLift before operating a motorized vehicle.
- Always exercise caution and care when using the ShoeLift.
- Improper use or installation of the ShoeLift could result in injury.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, consult your medical professional immediately.
- If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INFORMATION:

#### PREPARATION

- Select the appropriate size.


#### SHOELIFT APPLICATION

- Unfasten and loosen the top strap. (Fig. 1)
- While seated, place your uninjured foot over your opposite knee. Place the front of the ShoeLift cage over the toe of your shoe. (Fig. 2)
- Now pull the elastic cage toward the heel of the shoe and adjust the side straps of the cage and secure over the sole of the shoe. Pull the heel tab up over the heel of the shoe. (Fig. 3)
- Fasten the hook and loop strap securely over the top of shoe with the ProCare name facing out.

**CLEANING INSTRUCTIONS:** Hand wash in cold water using mild soap, or wipe with a damp cloth. Air dry.

### FOR SINGLE PATIENT USE ONLY.

~~NOT MADE WITH NATURAL RUBBER LATEX.~~

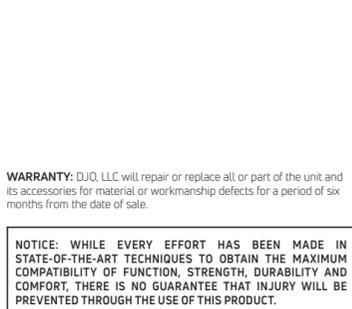**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOTICE:** WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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