DONJOY®

1

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

MyBabyStrap™ is a maternity support belt designed to reduce pain and discomfort associated with pregnancy:

- Lower back pain
- Pelvic pain
- Sacroiliac pain
- Postural instabilities
- Discomfort due to stretching and expansion of the abdomen

MyBabyStrap™ can also be used after birth for back support.

Soft-gond/semi-rigid designed to restrict motion through elastic or semi-rigid construction. Providing immobilization or controlled movement of the limb or body segment.

CONTRAINdications

If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
Any known allergies to the medical device materials.

WARNING & PRECAUTIONS

The product is intended for single patient use only.
If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact a medical professional immediately.
Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

- 1) Place the lumbar support around the waist, using the hand-loops.
- Fasten the lumbar belt using the hook closure on the front to support the abdomen.
- Make sure that the belt is properly centred over your spine.
- It is possible to change the position of the back pad (up or down) for additional support.
- 2) Pull the side strape to adjust the level of compression according to your needs and level of pain.
- 3) Fasten the side straps on the front part of the support using the hook closure.
- 4) For additional abdominal support, use the removable elastic strap.
- Position the elastic strap on the central part of the abdominal straps.
- Pull on both edges of the elastic strap until desired level of comfort and support.
- Fix them onto the abdominal straps using the hook and loop.
- 5) The Maternity Belt can be adjusted during pregnancy and after birth.
- In order to expand the lumbar support, reposition the lateral abdominal straps as follows:
- Unfasten the abdominal straps by detaching them from the internal hook and loop where they are fixed and slide them through the slits located close to the front of the lumbar support.
- Once desired length is obtained, secure the abdominal straps by attaching them to the hook and loop located close to the slits.
- Reposition the lumbar support around the waist and fasten it.

By modifying the position of the lateral abdominal straps, the Maternity Belt can be adapted to the waist circumference until the end of the pregnancy and also be used after birth.

CLEANING INSTRUCTIONS
Machine washable (max 40°C) with a mild detergent.
Hook and loop should be closed for washing.
Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean.
Do not iron.

COMPOSITION
Polyester, Polyamide, Polyurethane, Elastane, Cotton, Steel, Polycarbonate, Lurex, Viscose.

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

GARANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.
To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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Le produit est destiné à un usage unique par patient.
En cas de douleur, gonflement, changes in sensation or other unusual reactions occur while using this product, you should contact a medical professional immediately.
Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

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In order to expand the lumbar support, reposition the lateral abdominal straps as follows:

- Unfasten the abdominal straps by detaching them from the internal hook and loop where they are fixed and slide them through the slits located close to the front of the lumbar support.
- Once desired length is obtained, secure the abdominal straps by attaching them to the hook and loop located close to the slits.

• Reposition the lumbar support around the waist and fasten it.

By modifying the position of the lateral abdominal straps, the Maternity Belt can be adapted to the waist circumference until the end of the pregnancy and also be used after birth.

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Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean.

COMPOSITION

Polyester, Polyamide, Polyurethane, Elastane, Cotton, Steel, Polycarbonate, Lurex, Viscose.

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