ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS

The DonJoy Manuforce is intended to provide chronic, post-traumatic or post-operative support to the wrist joint and/or soft tissues. In particular, it may be suitable for use following wrist sprains, wrist tendinitis (tenosynovitis), mild osteoarthritis, rheumatic wrist pain or Carpal Tunnel Syndrome.

PERFORMANCE CHARACTERISTICS

Soft-good/ semi-rigid designed to restrict motion through elastic or semi-rigid construction.

FEATURES

- Removable, malleable palmar aluminum stay for stabilization of the wrist • Removable strap to adjust level of support • 3-D anatomically contoured knit provides comfortable, uniform support • Breathable material

CONTRAINDICATIONS

Do not use if you are allergic to any of the materials contained within this product.

WARNING & PRECAUTIONS

We recommend that the first application be made under the supervision of a healthcare professional. The support should only be used after consulting a health care professional if you suffer from skin diseases, injuries, systemic disorders, undercurrents, swellings of the hand/wrist, tendinitis or tenosynovitis, or if you have suffered soft tissue injuries, circulation or sensory disorders in the affected area. • The support should be used but not apply circulation. • If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should stop using the device and contact your healthcare professional immediately. • Do not use over open wounds. • If worn directly on the skin, the skin should be free from injuries, moisture, oil, creams, gel or similar residues. • Do not use the device if it is damaged and/or the packaging has been opened. **NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

1. Choose the support in Right or Right model according to the affected hand.
2. Pull the support over your hand and wrist, placing the thumb in the opening on the side.
3. For correct positioning, ensure that the hole of the silicon insert located on the interior of the support, covers the tuberosity on the lateral (external) part of the wrist.
4. The removable palmar aluminum stay can be adjusted to individual needs by bending it carefully.
5. If needed, attach the removable strap to the internal side of the support and wrap it around the wrist. The strap will provide additional support and stability, adjust it according to your needs. Avoid tightening the strap too much and loosen it immediately if you feel it is too tight.
6. Advice: When using the removable strap, avoid direct contact between the Hook & Loop of the strap and the knitted fabric, as this will damage the visual aspect of the support.
7. If you no longer need the aluminum stay for stabilisation, it can be removed. To remove the stay, hold the support by the top end and gently pull the aluminum stay out of its pocket.

USAGE INSTRUCTIONS

- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues when support is being used.
- The support should not be worn for more than 8 hours a day.
- With proper use and the observance of all instructions, no pressure points should occur.
- The manufacturer shall bear no responsibility for any changes to the product or incorrect usage.

CARE

- Hand wash in lukewarm water (30 °C) with a mild detergent
- Air dry (no place on radiators/heaters or other heat sources)
- Do not use bleach or chlorine, do not iron.

COMPOSITION

- Knitted material: Polyamid, elastin, polyester, natural rubber latex
- Insert: silicone
- Internal Cover: Terry cloth
- Strap: Polyamid
- Stay: Aluminum

LATEX CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.

FOR SINGLE PATIENT USE ONLY

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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