ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS: For Patellofemoral or Anterior Knee Pain resulting from excessive hip internal rotation and adduction. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: N/A

WARNINGS AND PRECAUTIONS:

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

NOTE: Application shown below is for the right leg strap, left leg strap is applied in the same manner.

- 1) Slide leg through the elastic calf band. Position the long end of the strap behind the medial femoral condyle (inside of the knee). The DonJoy logo should appear on the inside side of the knee. (Fig. 1)
- 2) Hold the elastic calf band in position, wrap the shorter "calf strap" around the upper calf muscle, and secure the strap with the Velcro tab. (Fig. 2)
- 3) Wrap the long strap around the thigh, above the knee. The strap will overlap, and cross, the front inner section of the thigh (as shown in diagram). Ensure that the overlapping portion crosses the thigh in a diagonal line, using the Velcro as a guideline. (Fig. 3)
- 4) Stand with the majority of your body weight on the unaffected (left) foot. With the right heel slightly touching the floor, torque (pull) the femur into external rotation by pulling strap across the thigh to the hip. Allow foot to rotate outward (toe-out). (Fig. 4)
- 5) Wrap the strap around the hip, so that it sits over the pelvis and greater trochanter, but below the iliac crest (on top portion of the hips). Anchor the strap with the Velcro tab. Ensure that the strap has been applied tightly. (Fig. 5)

RE-APPLICATION MARKERS

After the initial fitting session with the patient is complete, the medical professional can position the enclosed re-application markers on the brace to assist the patient with re-application.

- 6) Attach one of the small re-application markers to the overlapping medial (inner) portion of the strap by aligning the white line (below the DonJoy logo) on the marker with the white line on the strap. (Fig. 6)
- 7) Place the 2nd small re-application marker at which point the strap overlaps itself on the hips. (Fig. 7)
- 8) Align the large DonJoy marker near the belt buckle, aligning the hash marks with the midline of the body. (Fig. 8)

CONTENTS: Femoral Strap, 3 Re-Application Markers (2 Small, 1 "Belt Buckle"). Please do not apply the re-application markers until the strap has been properly fitted.

CLEANING INSTRUCTIONS: Hand wash in warm water with mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap can cause irritation or deteriorate material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY.

 **NOT MADE WITH NATURAL RUBBER LATEX.**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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