# AIRCAST®

## AIRGO PLUS / AIRSPORT +

### STABILISATION STRAP

### ZUSATZGURT

### SANGLE DE STABILISATION

DEUTSCH

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ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

#### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

#### INTENDED USE/INDICATIONS:

The Aircast Stabilization Strap is designed to provide additional optional support and stabilization to the ankle when using the Airgo Plus or AirSport+ brace. Soft-goods/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

#### CONTRAINDICATIONS:

Do not use if you are allergic to any of the materials contained in this product.

#### WARNINGS AND PRECAUTIONS:

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

#### APPLICATION - OPTIONAL STABILISATION STRAP

- To increase stability, you can extend the front cross-strap of AIRGO Plus/AirSport+ by using the stabilisation strap. Attach it to the cross-strap, with the arrow pointing to the right direction. (Fig. A)
  - Wrap the extended strap, centered around your Achilles tendon. (Fig. B)
  - Bring the strap forward, crossing over your forefoot. (Fig. C)
  - Run the strap under your foot. (Fig. D)
  - E Fasten the strap across the front. (Fig. E)
- If the stabilisation strap is too long, shorten it.

#### CARE

- Hand wash in lukewarm water (30 °C) with a mild detergent
- Air dry (do not place on radiators/heaters or other heat sources)
- Do not use bleach or chlorine, do not iron.

#### COMPOSITION

Polyester, Polyamide

**FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.**

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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MDSS GmbH
DJO FRANCE S.A.S.