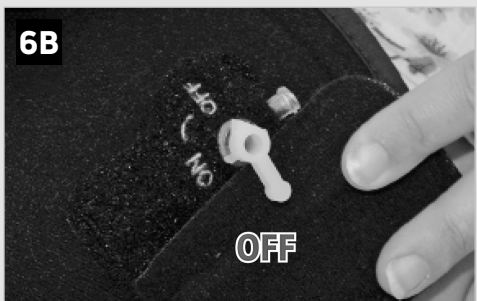
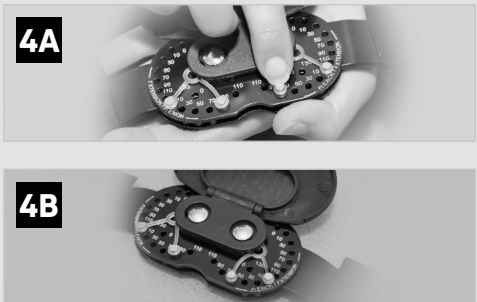
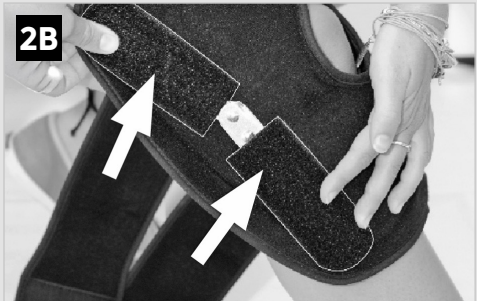
ENGLISH

Before using the device, please read the following instructions completely and carefully. Correct application is vital to the proper functioning of the device.

PLAYMAKER EVEREST™ / PLAYMAKER EVEREST™ PRO

- Playmaker Everest™ : Hinged knee brace
- Playmaker Everest™ Pro : Knee Brace with adjustable hinges

INTENDED USER PROFILE

- The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to:
- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

- Playmaker Everest™ - Indications requiring support and stabilization of the knee joint, such as: mild to moderate lateral ligament instabilities, meniscus lesions, patellofemoral pain, osteoarthritis, general knee pain or swelling, post-op use following knee surgery.
- Playmaker Everest™ Pro - Indications requiring support, stabilization and range-of-motion control of the knee joint, such as: mild to moderate ligament instabilities, meniscus lesions, patellofemoral pain, osteoarthritis, general knee pain or swelling, post-op use following knee surgery. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS

Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS

- We recommend that the first application be made under supervision of a health care professional.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.

Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

1) If using cold therapy option, pull the undersleeve over the knee (Fig. 1a). Place the Knee Cuff around the knee, with the patella opening centred on the knee cap. The valve at the top of the Knee Cuff (Fig. 1b).
2) Place the hinges into the pockets provided (Fig. 2a). Position on each side of the Knee Cuff, using the hook and loop (Fig. 2b).
Align the centre of the hinges with the centre of the knee cap. Ensure the 2 hinges are level on each side of the Knee Cuff.
If needed, the hinges can be formed to the patient's needs.

3) Attach the extension straps to the Thigh & Calf straps on the Knee Cuff.
Wrap the extension straps over the Knee Cuff and hinges to ensure a secure fit.
Use WITHOUT the hinges : During the treatment course, and upon advice of a health care professional, the Knee Cuff can also be used without the hinges and extension straps.
Secure the Knee Cuff by attaching the thigh Strap and the calf Strap using the hook and loop closures.
4) Adjustable Hinges
• To adjust flexion/extension, open the cap of each hinge, position the flexion/extension stops at the desired angle, and close the hinge cap (Fig. 4a).
• To avoid damaging the hinges, the flexion/extension stops must be positioned at the same angle on both hinges (Fig. 4b). **OPTION : COLD THERAPY & COMPRESSION**
For treatment combining Cold therapy and Compression, the Knee Cuff contains 1 Inflatable Pad, 1 Hand Pump and 2 Cold Packs.

INTENDED USE/INDICATIONS

- Localized external application of cold therapy and compression after surgery or injury, to help minimize inflammation, swelling, and pain.

CONTRAINDICATIONS - COLD COMPRESSION

DO NOT use this device on patients with: Raynaud's or other vasospastic diseases; diabetics; sickle-cell anemia, arteriosclerosis, cryoglobulinemia, hypercoagulable clotting disorders or other peripheral vascular diseases causing ischemia or poor local circulation; compromised local circulation; local tissue infection; cold allergy, cold hypersensitivity, previous cold injury.

APPLICATION INSTRUCTIONS - COLD COMPRESSION

- Prior to use, detach Cold Pack from Knee Cuff and place in freezer for 4 hours minimum. (Domestic deep freeze temperature between -12°C & -20°C).
- Once cold pack is cold / frozen, attach the Cold Pack onto the Knee Cuff, using the hook and loop (Fig. 5). Feel Cold Pack and ensure temperature is adequate before use.
- WARNING! Be sure to place a barrier/cloth between your skin and the Cold Pack (undersleeve). (See "WARNING & PRECAUTIONS"). Cold Pack temperature should not be lower than 5°C.
- Apply the Knee Cuff and hinges as per Application Instructions 1-4.

6) Screw the pump to the Knee Cuff, turn the valve to 'ON' position (Fig. 6). Inflate the Air Cell to the desired level of compression (Fig. 6a). Once the desired inflation level has been reached turn the valve to 'OFF' and remove the pump (Fig. 6b).
• WARNING! Do not inflate and tighten the Knee Cuff excessively. Ensure level of compression is not too tight so as not to restrict blood or fluid flow. (See "WARNING & PRECAUTIONS")
• WARNING! Upon instruction of a Health Care Professional: Apply the Knee Cuff to injured area for time recommended by a Health Care Professional. Do not exceed 20-30 minutes period of application. Alternate with adequate rest periods.

If needed, renew application using second Cold Pack. Replace first Cold Pack in freezer until needed. Do not sleep with the Knee Cuff on.
7) To deflate, screw the pump to the Knee Cuff, position the valve on "ON", and press the button at the base of the pump to release the air (Fig. 7).

WARNINGS AND PRECAUTIONS - COLD COMPRESSION

WARNING! THIS DEVICE CAN BE COLD ENOUGH TO CAUSE SERIOUS INJURY. SERIOUS ADVERSE REACTIONS AND SAFETY HAZARDS MAY OCCUR WHEN USING THIS DEVICE.

USE OF KNEE CUFF WITH INFLATABLE PAD & COLD PACK :

- A health care practitioner must state how long and how often the device should be used and the length of breaks between uses.
- Extreme care must be taken when using any cold therapy system. Any cold products may cause injuries, e.g. burns, blisters, frostbite, swelling, discoloration and skin or tissue necrosis if improperly used or monitored.
- WARNING! DO NOT** inflate and tighten the Knee Cuff excessively. Ensure level of compression is not too tight so as not to restrict blood or fluid flow.
- WARNING! This device can be cold enough to cause serious injury, including tissue necrosis.** You must be able to check your skin condition under the cold pad frequently (at least every hour). DO NOT use this device if you cannot check your skin condition frequently (at least every hour). Check for increased pain, burning, numbness, tingling, increased redness, discoloration, itching, increased swelling, blisters, irritation or other changes in skin condition under the cold pad or around the treatment area. If you experience any of these conditions, immediately discontinue use of this device and contact your physician.
- WARNING! Application of the cold pads directly on the skin may result in serious injury, including tissue necrosis.** DO NOT let any part of the cold pad touch your skin. ALWAYS use with a barrier/cloth between your skin and the cold pad.
- Keep the product away from children.
- All Knee cuff ingredients are non-toxic. They are, however, not intended for ingestion.
- Do not heat the Cold Packs (microwave, oven, hot water, etc...)
- Do not use on open wound(s) and/or sensitive skin.
- Do not use this product if it is damaged or defective.
- Do not use this product in any way that may damage its integrity.
- Do not sit or lay over the Cold Pack or fall asleep with the cold pack on the skin.
- If contents get in contact with eye or skin, rinse with water immediately.

FRANÇAIS

anesthetics, as well as patients taking hypnotics, anxiolytics, or antidepressants, must be monitored frequently during use of this device. These patients may not be able to perceive pain, burning, numbness, tingling or decreased sensation and may be susceptible to injury. Discontinue cold therapy immediately at the first sign of cold injury.

CARE - KNEE CUFF

- Wrap may be hand washed and air dried.
- Gel inserts may be cleaned with water, moist towel or sponge.

COMPOSITION

Knee Cuff
Inflatable Pad: Polyester, nylon
Pump: Silicone, Polyethylene, PVC, ABS Resin (thermoplastic polymer), synthetic rubber and steel wire
Cold Pack
Gel: propylene glycol, sodium carboxymethyl cellulose, water.
Fabric: cotton, nylon
Hinges : Aluminium, polyethylene, nylon
Undersleeve : Polyester

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.

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